

Battery Device Adapter Manual

Please read this entire manual before operating this device.



800-322-0956
www.ablenetinc.com





To Use the Battery Device Adapter

1. Place the copper disk between the battery and the battery contact in the toy. Close the battery compartment cover. You may need to remove a portion of the cover to allow the wire to fit through. Use a file to make a small notch in the battery compartment cover.
2. Plug a switch into the Battery Device Adapter.
3. Turn the toy's on/off control to the "on" position. Now the switch can be used to turn the toy on or off.

Equipment Trouble

If you followed the instructions and the device fails to operate, check the following:

- Does the battery operated toy work independently of the Battery Device Adapter if removed? If not, a problem may exist with the toy or device. Check the batteries in another toy to ensure the batteries work.
- Does the switch work with other battery operated toys or control units? If so, a problem may exist with the Battery Device Adapter.

Still not working?

If the AbleNet Battery Device Adapter still refuses to work, it may be defective. **DO NOT ATTEMPT TO TAKE IT APART.** Call an AbleNet customer service representative at (800) 322-0956 or visit ablenetinc.com for assistance.

Warranty and Repair

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

For warranty and repair outside the United States please contact your local AbleNet reseller.

For more information on AbleNet products and services, contact: AbleNet, Inc. Minneapolis/St. Paul, MN | 800-322-0956 | www.ablenetinc.com