

# Battery Device Adapter Manual

*Please read this entire manual before operating this device.*



800-322-0956  
www.ablenetinc.com





## To Use the Battery Device Adapter

1. Place the copper disk between the battery and the battery contact in the toy. Close the battery compartment cover. You may need to remove a portion of the cover to allow the wire to fit through. Use a file to make a small notch in the battery compartment cover.
2. Plug a switch into the Battery Device Adapter.
3. Turn the toy's on/off control to the "on" position. Now the switch can be used to turn the toy on or off.

## Equipment Trouble

If you followed the instructions and the device fails to operate, check the following:

- Does the battery operated toy work independently of the Battery Device Adapter if removed? If not, a problem may exist with the toy or device. Check the batteries in another toy to ensure the batteries work.
- Does the switch work with other battery operated toys or control units? If so, a problem may exist with the Battery Device Adapter.

## Still not working?

If the AbleNet Battery Device Adapter still refuses to work, it may be defective. **DO NOT ATTEMPT TO TAKE IT APART.** Call an AbleNet customer service representative at (800) 322-0956 or visit [ablenetinc.com](http://ablenetinc.com) for assistance.

## Warranty and Repair

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

For warranty and repair outside the United States please contact your local AbleNet reseller.

For more information on AbleNet products and services, contact: AbleNet, Inc. Minneapolis/St. Paul, MN | 800-322-0956 | [www.ablenetinc.com](http://www.ablenetinc.com)