
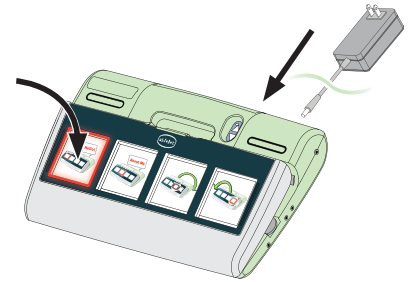


Thank you for purchasing the FL4SH scanning communicator!
You're only minutes away from successful communication.
Please take a moment to review both sides of this guide.

1 Power Up and Follow the Lights

1. Turn FL4SH ON.*
2. Activate the first message location (flashing). FL4SH will begin walking you through the process of recording your first overlay.
NOTE: To replay any message, press that location again.


*FL4SH ships with a partially charged battery and may be used while plugged-in. Full charge will take approximately 2 hours. (BAT. light turns green.)

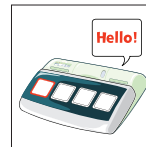

2 Access the Surface Panel and Overlays

To open the Surface Panel


1. Press the Release Button (highlighted at right).
2. Firmly, pull silver Surface Panel towards you.
3. Remove the first Overlay stored inside FL4SH.

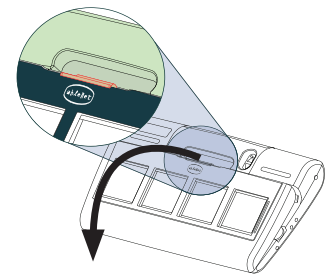

3 Access the Control Panel and Instructions

1. With the Surface Panel open, locate the two "PRESS" indicators on the green housing.
2. Using your thumbs, press on the latches (highlighted at right) to release the Control Door.
3. Lift the Control Door towards back of device.


NOTE: Programming Instructions are under the Control Door.

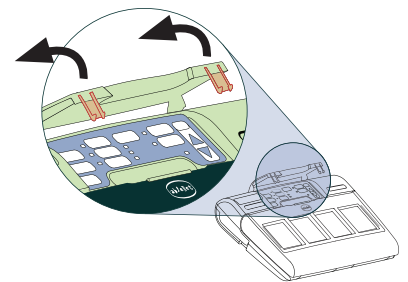


4 Replace the Master Overlay

1. Remove the Keyguard and Master Overlay.
2. Replace the Master Overlay with the overlay from Step 2.
3. Replace the Keyguard.
4. Turn both the Master Overlay and this guide over for instructions on recording your first Overlay.

NOTE: To remove the Keyguard, pull forward and lift up on the dark blue Keyguard near the AbleNet logo.

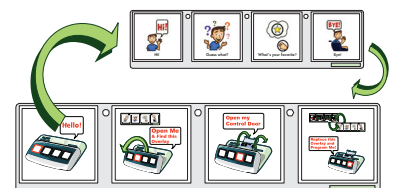

Recording Messages with an Overlay

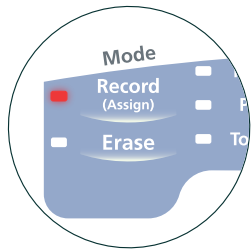


1. Press "Record" under the Mode column.

You have now entered Record Mode.




2. Press "Message" under the Item column.

You have now selected the Item to Record.

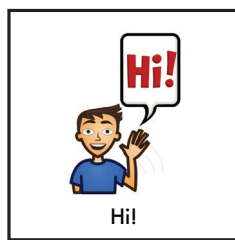

3. Press and hold the location you wish to record. Begin speaking at the beep. Release when finished.

Repeat for additional messages.

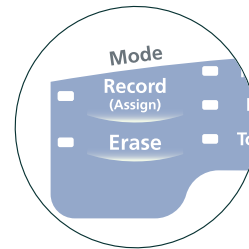

4. Select "Record" from Mode column to exit Record Mode.

Next Steps

For complete programming instructions and Scanning Activities, see the FL4SH Manual CD-ROM located inside the 3-Ring Binder.


For a brief overview of FL4SH's scanning features, see the Scanning Configuration Card located inside the 3-Ring Binder.

Register your FL4SH and receive a free Game Overlay. See the Registration Card or visit www.ablenetinc.com for details.


FL4SH Includes the Following Accessories




CD-ROM manual & activities
(included in Binder)

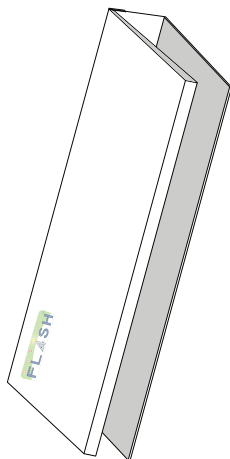

3-Ring Binder

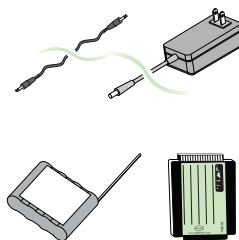

- Toy/Appliance Cable
- A/C Adapter

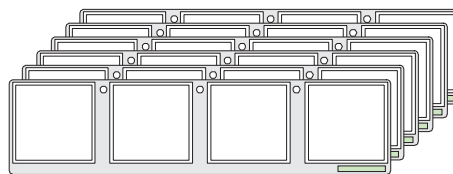

Overlay Sleeves (14)

FL4SH Deluxe Kit also Includes:

2 Jelly Bean® switches
Additional 8-Minute Module with 12-Overlay Packet

AbleNet Travel Strap - Shoulder Mount
AbleNet Tote Bag
4 AA batteries

Additional Accessories available on-line at www.ablenetinc.com