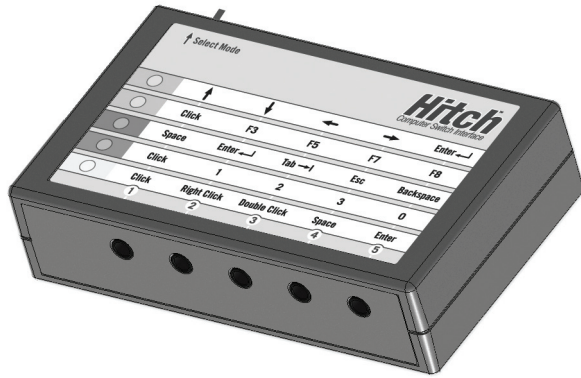
## Quickstart Guide:

Hitch™ computer switch interface

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www.ablenetinc.com

### Tips & Tricks

The F3, F5, F7, and F8 functions in the Orange row are especially useful with some software programs. For a list of compatible switch software and their functions, search "hitch" under Support at [www.ablenetinc.com](http://www.ablenetinc.com).

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### Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on products. For full warranty details, visit [www.ablenetinc.com](http://www.ablenetinc.com).

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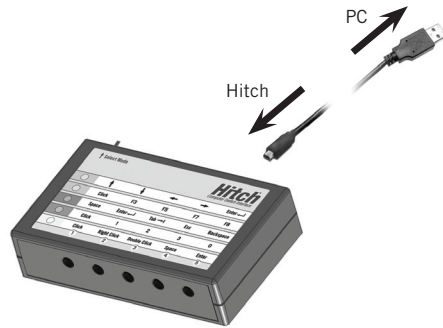
### For more information on AbleNet products and services:

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Connect Hitch using the included USB cable.

All five LEDs will light briefly when connected.

NOTE: Hitch will work with any Mac or Windows-based computer with an available USB port.

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
## 2 Choose Functions

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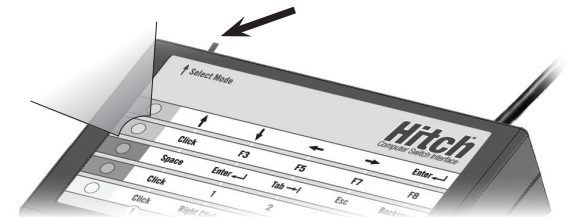Press the "Select Mode" button to cycle through the available modes by row.

Hitch will remember its previously selected mode when unplugged or moved between computers.

NOTE: Only one complete row of functions may be selected at a time.

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
## 3 Connect Switches

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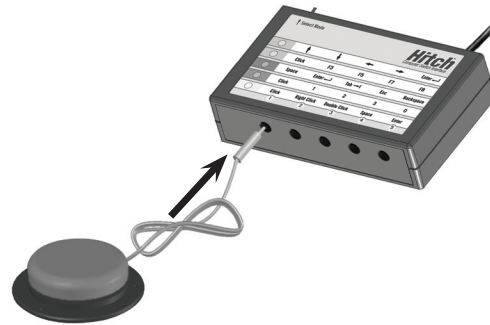Connect switches to the desired switch jacks on the front of the device.

Hitch can support up to five 1/8-inch (3.5 mm) switches at one time.

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Press any switch to send the selected function to the computer.

The computer will respond as though the corresponding keyboard or mouse button has been pressed.

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© 2009 AbleNet, Inc., All Rights Reserved
© 2009 AbleNet, Inc. Alle Rechte vorbehalten tous droits réservés