
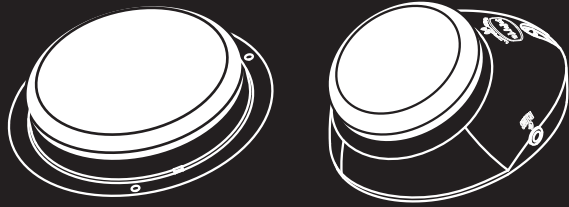
Quickstart Guide: BigBeamer™ and JellyBeamer™ Wireless Transmitters

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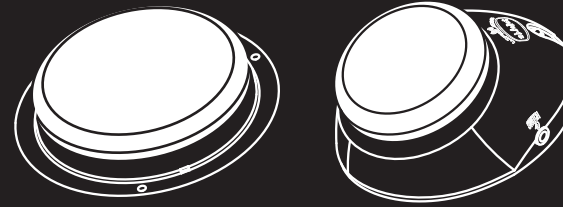
ablenet®

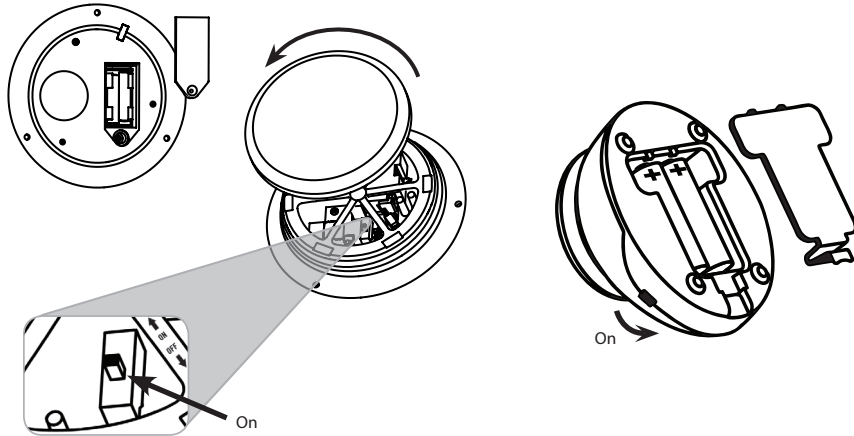www.ablenetinc.com

1. Insert two batteries into the Transmitter as shown on housing.

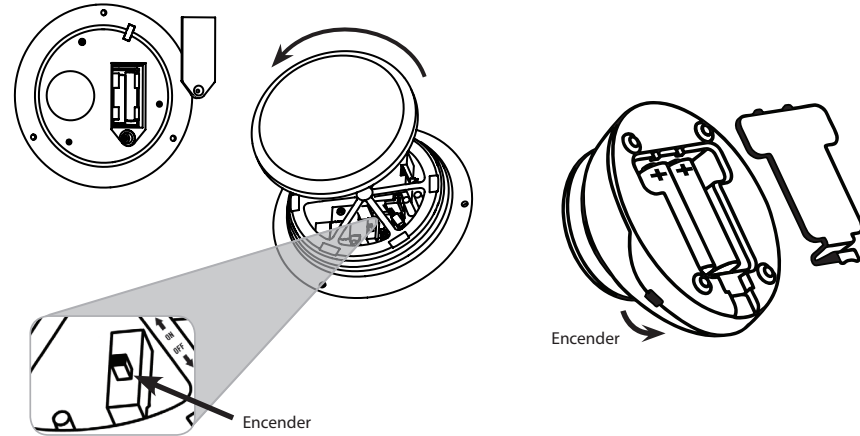
Big Beamer Transmitter uses AAA batteries. Jelly Beamer uses AA batteries.
When batteries are low, the device's light will flash three (3) times.

2. Slide the power switch* to the "On" position.

The Transmitter's light will activate.
*Big Beamer's power switch is located beneath the switch top.

3. Link Transmitter with linkable™ Receivers or other linkable products.

See any linkable product manual for linking instructions.

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Additional Information

Visit www.ablenetinc.com for mounting options. Big Beamer and Jelly Beamer Transmitters have an effective range of up to 30 feet. To clean, wipe surfaces with a slightly damp cloth.

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com. For more information on AbleNet products and services, contact: AbleNet, Inc. Minneapolis/St. Paul, MN | 800-322-0956 | www.ablenetinc.com

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