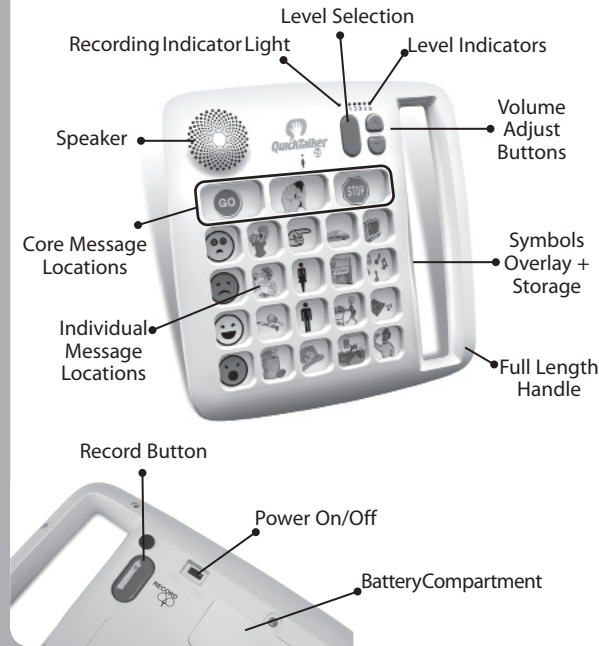
Quickstart Guide
 QuickTalker™ 7
 QuickTalker™ 12
 QuickTalker™ 23

www.ablenetinc.com

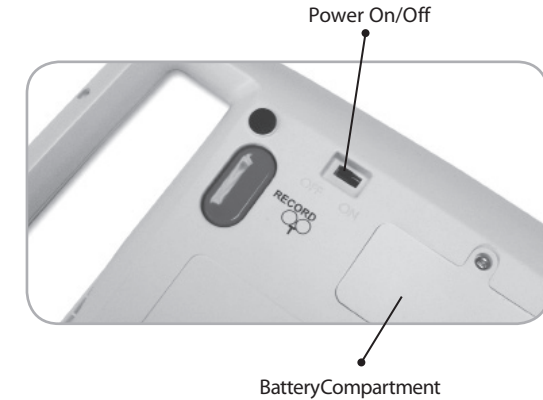
Key Features

Step 1 - Powering QuickTalker™

Powering QuickTalker™


Turn the device to the back side. Insert 4 AA batteries into the device where shown. Use the On/Off switch to power the device.

Step 2 - Select Level

Select Level

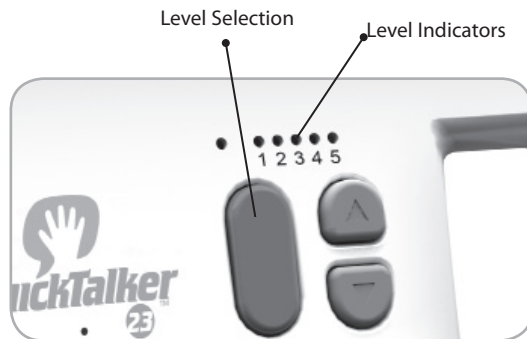
Select LEVEL (1-5) by pressing the red button on the front of the device.

Step 3 - Recording Messages

a) Entering Record Mode

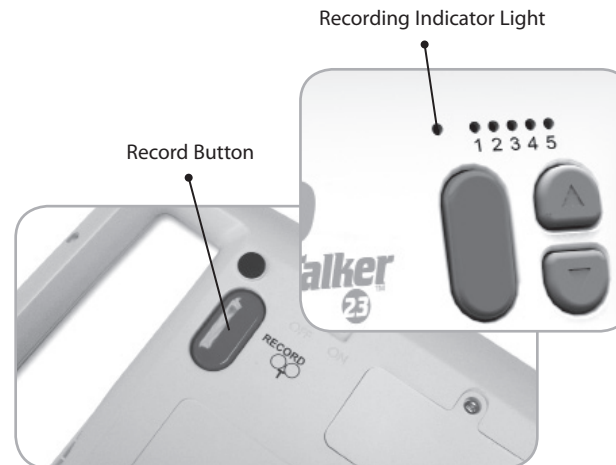Press and hold the RECORD button on the back of device until the red light is on, then release.

b) Recording Messages

Press and hold the desired message location, begin speaking after the beep, then release. Repeat for other locations.

Tip! *Speak 6-8" from the microphone for loud, clear sound.*

c) Exiting Record Mode

Press and release the RECORD button. Lights turn off. Note: The device will automatically exit Record Mode after 20 seconds of no activity.

Step 4 - Playing a Message

Playing a Message

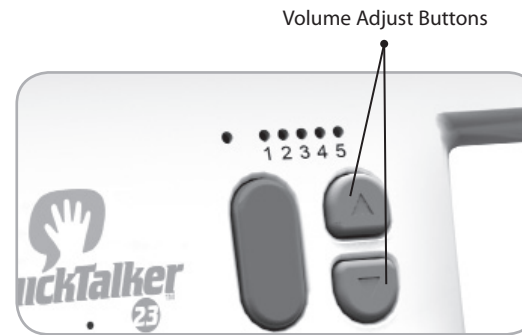
Press the desired location to play a message.

Note: Pressing another location will stop the current message.

Adjusting Volume

Press the gray arrow buttons to adjust the volume.

Message and light function

When a location is pressed the record light will flash yellow if the location is blank; red if recorded.

Warranty and Care

Cleaning and Care

To clean wipe with a slightly damp cloth.

Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

More Information

You can find warranty, guarantee and return/repair information online.

For more information on AbleNet products and services, or for contact:

AbleNet, Inc.

Mpls./Saint Paul, MN 55113

800-322-0956

www.ablenetinc.com

Erasing Messages

Single Location - Enter Record Mode, then quickly press and release the location.

Level - Press and hold RECORD button for 10 seconds and all messages on current level will be cleared. Core messages will remain.

Master Clear - Press and hold RECORD button for 30 seconds and ALL QuickTalker messages will be cleared.

Record Button

Advanced Features

Core messages

The three core messages will be the same for all levels. The last recorded messages will be played.

Locking the Controls (RECORD & LEVEL)

Press and hold the red RECORD and LEVEL buttons simultaneously for 5 seconds. All lights will flash twice. Repeat to unlock controls.

Record Button

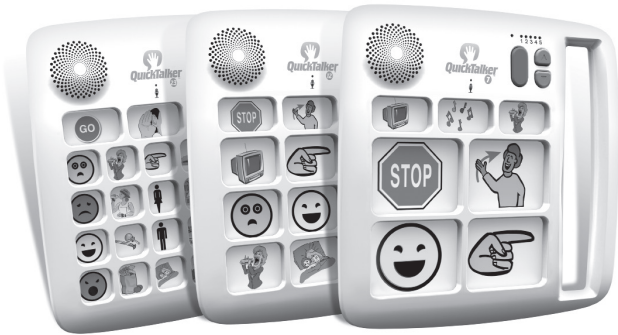
Level Indicators


Guide de mise en route rapide

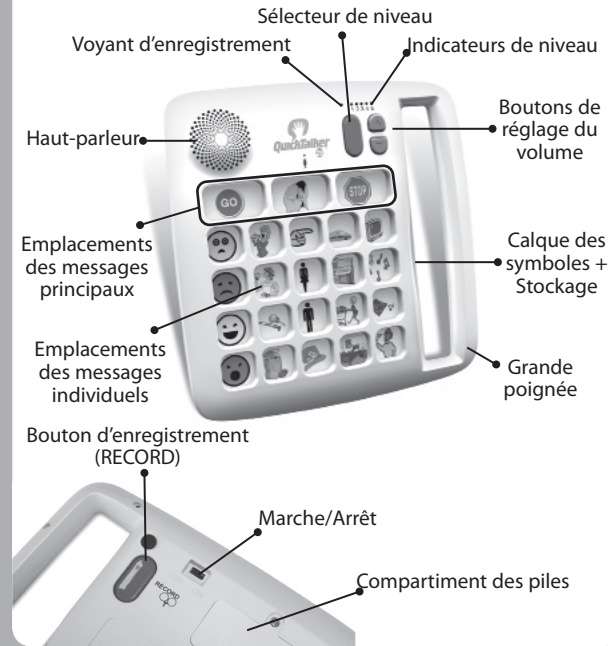QuickTalker™ 7
QuickTalker™ 12
QuickTalker™ 23

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