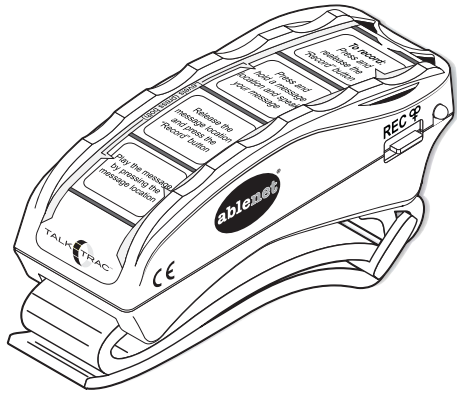
Quickstart Guide

TalkTrac™ Wearable Communicator

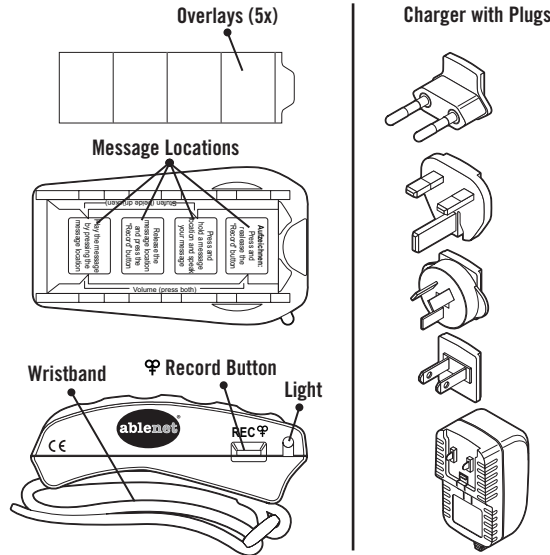
- Eight - 10 second messages
- Two levels of 4 messages
- Two volume levels
- Rechargeable


www.ablenetinc.com

Overview

NOTE: The wristband can be replaced with standard wristbands.

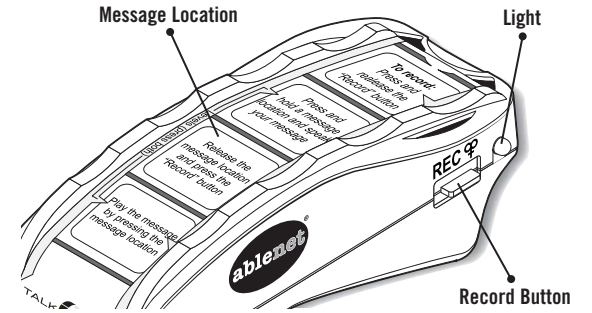Record and Play Messages


To Record a Message

1. Press and release the REC button (red light).
2. Press and hold a message location (light flashes red) and speak message.
3. Press and release the REC button (light off).

To Play a Message

Press and release the message location.

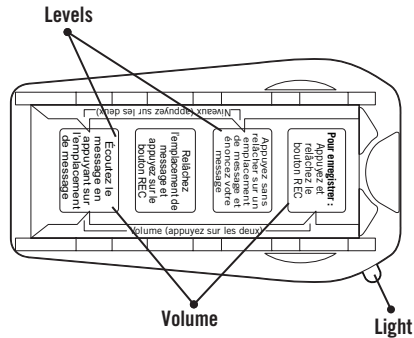Volume and Changing Levels


To Change Volume

Press both volume locations shown.

To Change Levels

Press both level locations shown (green light will flash the level, 1 or 2).

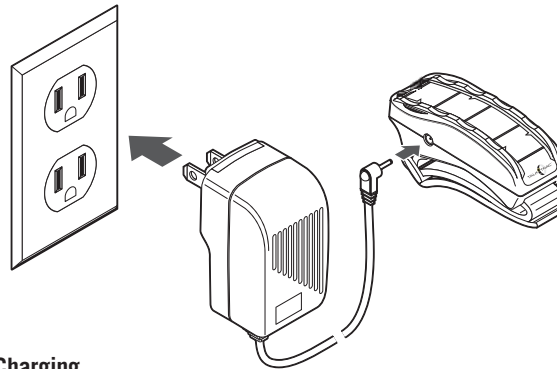Charging and Low Battery

Charging

Charge each night to maintain a full charge.

NOTE: Charge for 24hrs before first use.

Low Battery Warning

TalkTrac will flash red multiple times after a message play. The battery level is getting low and should be charged soon.

Overlays / Warranty and Contact

Overlays

Additional overlays and can be created using Mayer-Johnson® Boorkmaker or by downloading a blank template from AbleNet's website.

Warranty/Repair/Return Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

For more information on AbleNet products and services:

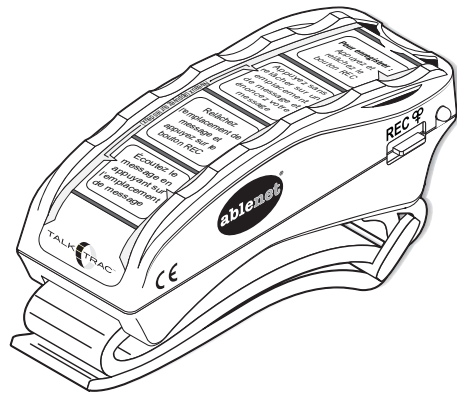
AbleNet, Inc.,
 Minneapolis/St. Paul, MN
 800-322-0956 (US & Canada)
 Internet: www.ablenetinc.com

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