

Quickstart Guide

DynaVox V/Vmax/M3 Mount for TrackerPro™




www.ablenetinc.com

What's Included

Kit Contents


You have received a plate with a TrackerPro™ mount attached, 2 – short hex screws, 2 – long hex screws, and 2 hex keys (1 small and 1 large).

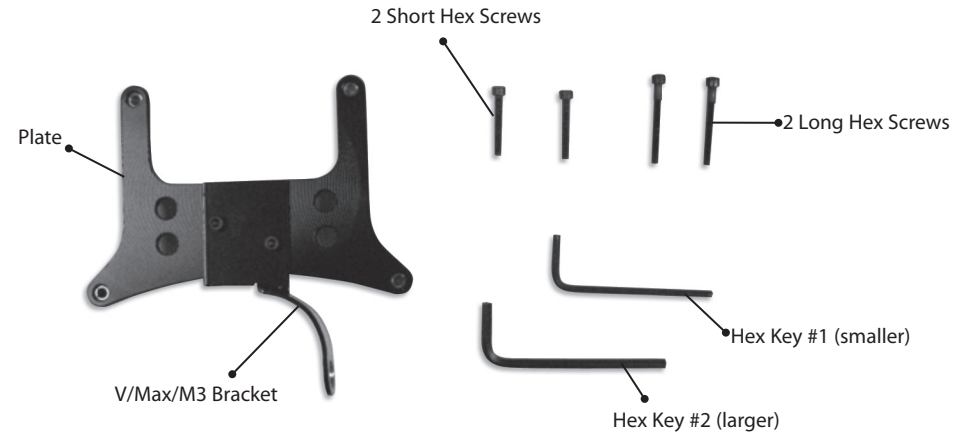

Step 1

Remove Existing Mount


Prepare TrackerPro™ for mounting by removing the existing mount with the dual lock using the large hex key #2. The button screw will be tight, as the threads have Threadlocker on them. Don't remove the Threadlocker, as it will hold the screw in place when TrackerPro is re-mounted.

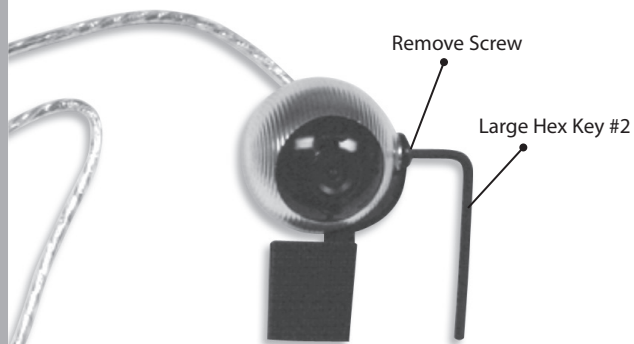

Step 2

Mount TrackerPro™ on the V/Vmax/M3 bracket.


Using hex key #2, tighten the large button screw until snug. You should be able to rotate the TrackerPro™ and have it hold its position.

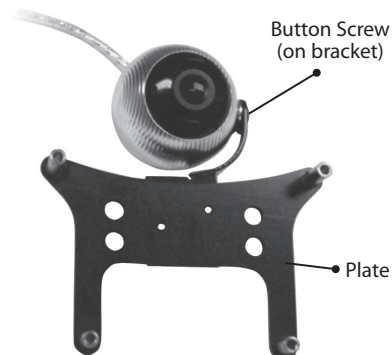

Step 3


Prepare the DynaVox unit.

Remove the four centrally located (two front, two rear) hex socket screws from the top of the DynaVox unit, using the small hex key #1.

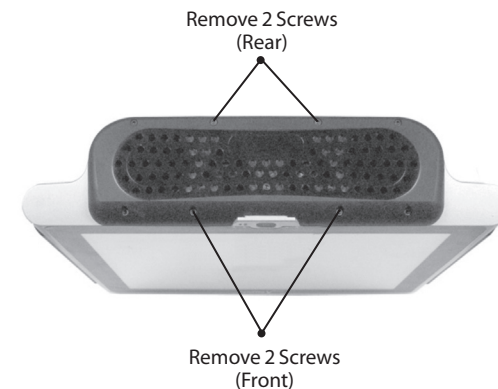

Step 4


Slide the stand-offs on the bottom of the DynaVox mount into the four screw holes. Use two longer screws in front, shorter two in the rear – until the plate is parallel and about 1/8th of an inch above the DynaVox unit.

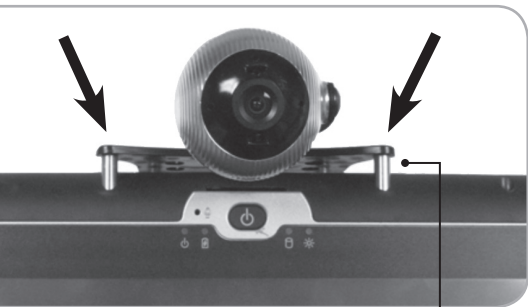

1/8" Gap

Step 5


Screw plate to DynaVox unit.
Place the 2 longer screws in the front stand-offs and the 2 shorter screws in the rear and screw them in with the hex key until they are snug.

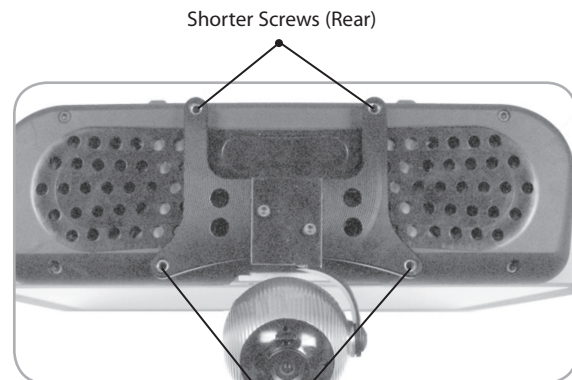

Longer Screws (Front)

Step 6

Connect TrackerPro™ to DynaVox unit.
Plug the USB connector on the TrackerPro into the USB port on the DynaVox unit, and you are ready to go!

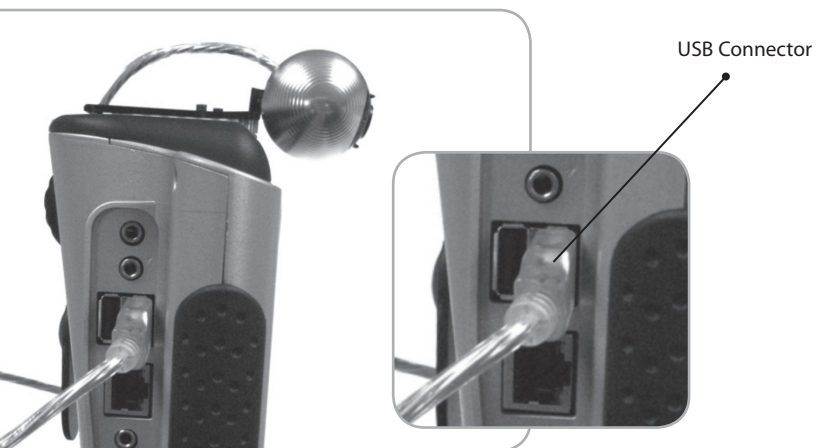

Warranty and Care

Cleaning and Care
To clean wipe with a slightly damp cloth.

Warranty
AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

More Information
You can find warranty, guarantee and return/repair information online.

For more information on AbleNet products and services, or for contact:

AbleNet, Inc.
Mpls./Saint Paul, MN 55113
800-322-0956
www.ablenetinc.com

PN 900-80220 Rev. A (11/10)


www.ablenetinc.com