
# QuickStart Guide

## VideoBrix™ Single Message Video Communicator


- Easy single-message video communication
- Up to 90 seconds recording time
- Rechargeable

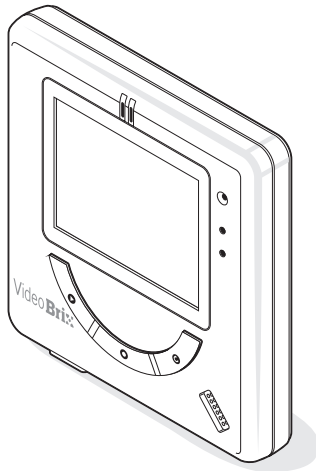
www.ablenetinc.com

### 1. Product Overview & Charging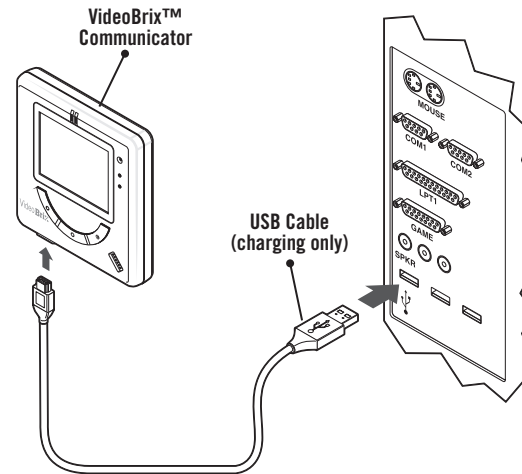

Battery life: 20 minutes active use. Connect to a computer to charge for 2 hours before use. A wall outlet may be used with an adapter (not included).

### 2. Power On / Playing Message

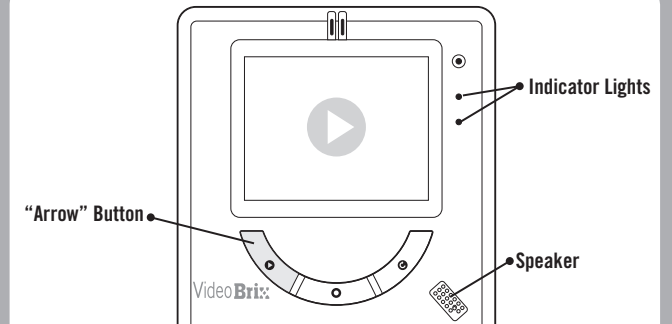
#### To Power On and Play an Existing Message

Press the "Arrow" button to power on VideoBrix and play an existing message. After the message plays, the screen will display the "Repeat" symbol.

Note: If you are using VideoBrix for the first time (no existing message), the "Record" symbol will display when the unit is powered on. See Step 4 to record your first message.


### 3. Replaying a Video Message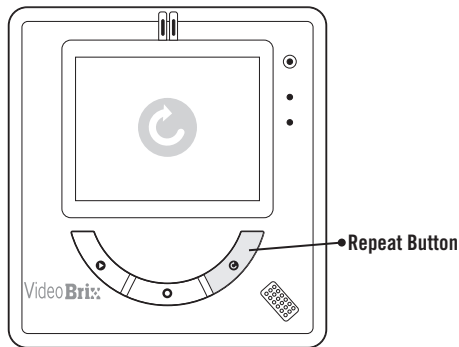

#### Replaying a Message

Press the "Repeat" button to replay a message within 30 seconds of playing it for the first time. The on-screen icon will remind you which button to press. The unit will go into standby mode after 30 seconds. Press the "Arrow" button to power on and play your message (see step 2).

### 4. Recording a New Message

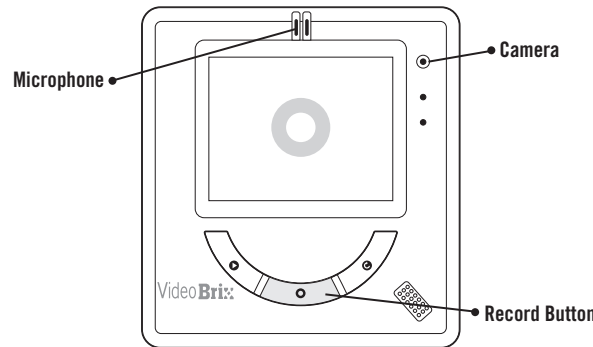#### Recording (or Re-recording) a Message

Press the "Record" button to begin recording a video message using the built-in camera and microphone. Press the "Record" button again to stop recording.

Note: Recording a new message will erase any existing message.

### Warranty and Contact Information

#### Warranty/Repair/Return Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two years from the date of purchase. Warranty in full text is available at [www.ablenetinc.com](http://www.ablenetinc.com).

#### For more information on AbleNet products and services:

AbleNet, Inc.  
Minneapolis/St. Paul, MN  
800-322-0956 (US & Canada)  
Internet: [www.ablenetinc.com](http://www.ablenetinc.com)

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