



Quickstart Guide: AbleNet Wireless Pager

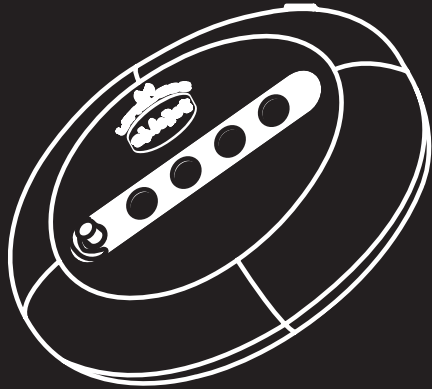


ablenet

www.ablenetinc.com

1 Insert Batteries and Turn On

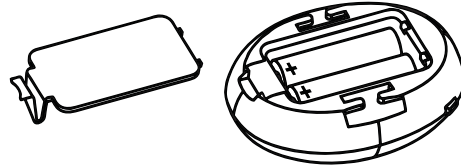


1. Insert two AA batteries into Pager as shown.

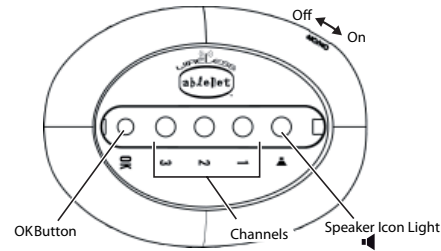


2. Slide the power switch to the "On" position. The Pager's light(s) will activate briefly.

2 Link Pager & Transmitter

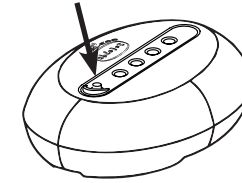

Select Channel,
Press & Hold

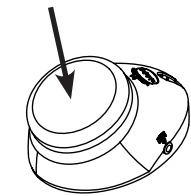

Press &
Release



NOTE: This pager only works with select AbleNet products. See Advanced Features below for details.

1. With both Pager and Transmitter/ECU turned on, press and release the Pager's **OK** button to select Channel 1, 2, or 3.
2. Press and hold **OK** button until the  light turns green (about 5 seconds).
3. Activate any Transmitter/ECU. *The  light will flash (2X) when a link has been established.*
4. Repeat for additional channels.

Receiving Pages

When a page is received, the Pager will flash, vibrate, and beep. The LED will indicate which channel is being paged. This will continue until the page is snoozed, acknowledged, or cleared with the **OK** button.

To Snooze a Page:

Press **OK** button one (1) time to enter Snooze (silence) mode.

The appropriate LED will flash every 2 seconds, and the Pager will beep and vibrate every 5 minutes until the page is cleared.

To Acknowledge a Page:

Press **OK** button two (2) times to send acknowledge signal.

To Clear a Page:

Press and hold **OK** button for 5 seconds.

The corresponding LED will turn off.

Advanced Features

To Mute/Un-mute the Pager:


When no pages are active,

1. Press **OK** button to highlight 
2. Press and hold **OK** button for 5 seconds to toggle.

Mute = Chirp

Un-Mute = Long Beep

Unlink Transmitters

1. Press **OK** button to select channel.
2. Press and hold **OK** button for 10 seconds until  light flashes red.

Compatible Transmitters

This Pager will only work with linkable AbleNet Wireless devices including the Jelly Beamer, Big Beamer, and Sway Wireless.

Low Battery Warning


When batteries are low, the device's  light will flash three (3) times.

Tricks, Tips, and Warranty

Attach Belt Clip

To attach included Belt Clip,

1. Unfasten Velcro ends.
2. Thread ends through the Belt Clip Slots.

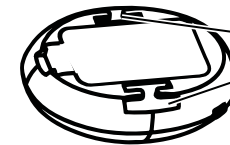

Belt Clip Slots

Cleaning

To clean, wipe surfaces with a slightly damp cloth.

Warranty Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com. For more information on AbleNet products and services, contact:

AbleNet, Inc.

Minneapolis/St. Paul, MN | 800-322-0956 | www.ablenetinc.com