


Quickstart Guide

Responsible Personal Alert System

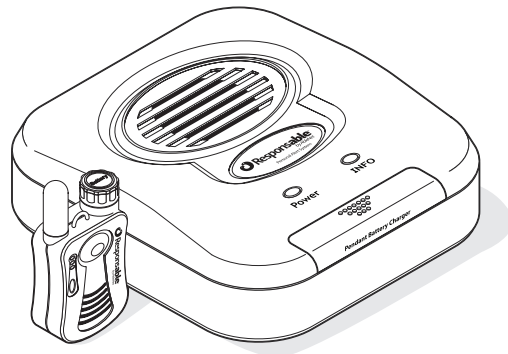


Distributed by AbleNet




www.ablenetinc.com

Inserting Batteries

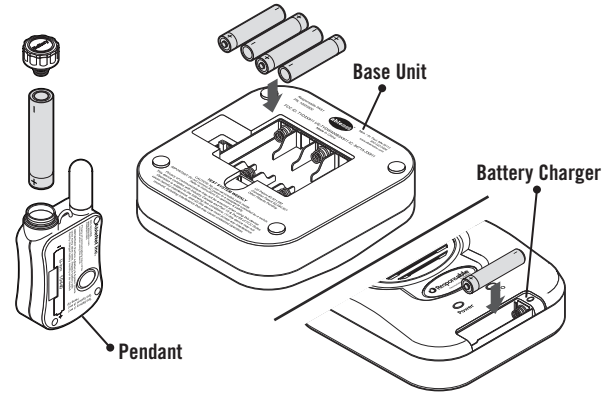


1. Insert 4 AA batteries into Base Unit as shown on device.
2. Charge unused Pendant battery in front of Base Unit as shown.
3. Insert battery into Pendant as shown.

Connecting Telephone Lines & Power

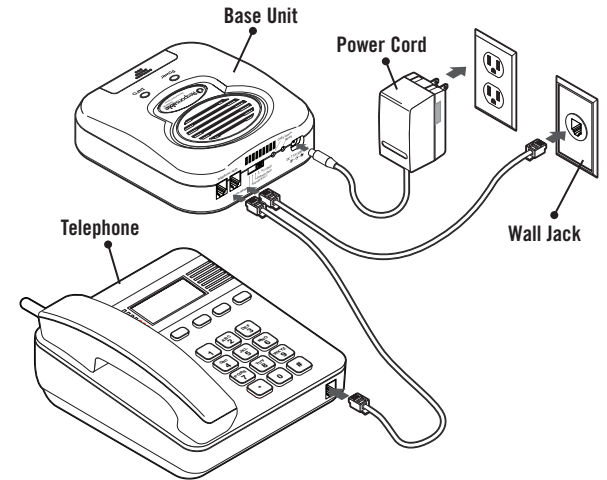

1. Connect Base Unit to power outlet.
2. Connect Base Unit to telephone wall jack.
3. Connect existing telephone to Base Unit.

Establishing a Phone Connection

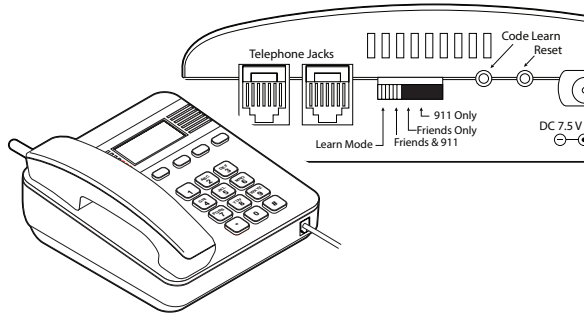

Enter Learn Mode


1. Dial any outside line to establish a connection.
 - A personal mobile phone number is best. If you call a friend, tell them to stay on the line while Responsible is being programmed.
 - This step ensures that the device does not attempt to dial each number as you program it.
2. Move mode switch on Base Unit to Learn position.
3. Enter programming code: 1234 then #.

Programming Friends Numbers



1. In order of priority, dial friend's telephone number then press # to accept.
 - Include area code, 1 for long distance, or 9 for outside line as-needed.
2. Repeat for numbers 2, 3, and 4.
 - Press # at any time to skip entering additional numbers. Hang up when complete.
3. Exit Learn Mode by moving slider to "Friends Only".

Making a Test Call

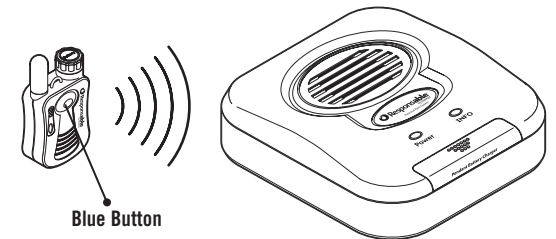


1. To avoid dialing 911 during the test, move mode switch on Base Unit to "Friends Only" position.
2. Press and Hold Blue button on Pendant for 2 seconds.

The Base Unit will dial Friend #1. If the call is answered, use the pendant to speak, holding it like a traditional phone.

3. Press and Release Gray button on Pendant to hang up.

Calling Out for Help

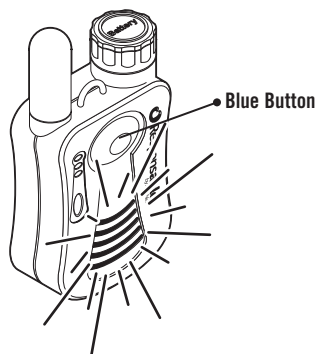

1. To call for help, Press and Hold the Pendant's Blue button for 2 seconds.
 - Responsible will dial the emergency number(s) based on the calling mode selected.
2. Hold the Pendant close to your mouth like a traditional phone while speaking.

Your Emergency Call




Note: Each emergency call made with Responsible must be "acknowledged" by the recipient. To do this, Responsible will prompt the recipient to press 5 on their keypad.

If the first contact in your list does not answer or acknowledge your call, Responsible will automatically dial the next number programmed.


Depending on the calling mode selected, Responsible can dial 911 if none of your emergency contacts answer or acknowledge your call.

Ending a Call

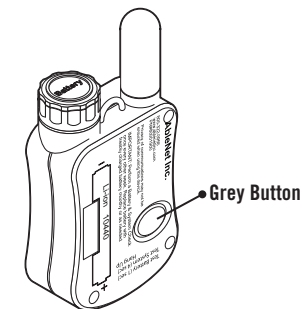


1. To hang up, Press and Release the Gray button one time.
2. The recipient can also end the call by pressing 9 on their keypad (this is the preferred method) or by simply hanging up.
 - When the recipient hangs up without pressing 9, Responsible will emit a tone for several seconds until the line is deemed inactive before eventually hanging up automatically.

Answering an Incoming Call

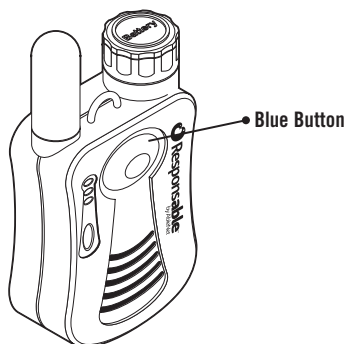

To answer an incoming call, Press and Release the Pendant's Blue button one time.

Phone Calling Modes

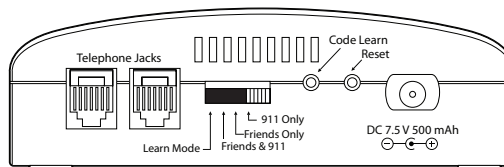

Use the slider switch on base unit to select mode of operation:

- Learn Mode – Used for programming telephone numbers into memory.
- Friends & 911 – Responsible dials all Friends, in order entered until call is accepted. If not accepted, Responsible dials 911.
- Friends Only - Responsible dials all Friends, in order entered until call is accepted. If not accepted, Responsible hangs up.
- 911 Only – Responsible dials 911.

Warranty and Contact Information

Additional Pendants and Emergency Wall Communicators are also available from AbleNet. Responsible can be paired for use with up to 4 Pendants or Wall Communicators at one time.

Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

AbleNet, Inc.

Mpls./Saint Paul, MN 55113
800-322-0956
www.ablenetinc.com



www.ablenetinc.com