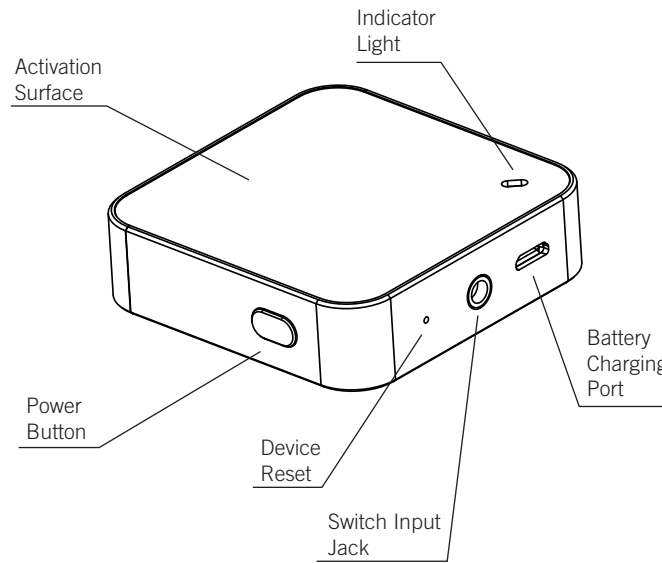
## Quick Start Guide Mini Beamer™ Transmitter & Receiver

www.ablenetinc.com  
1-800-322-0956

PN 9800021 (07/13 Rev. A)

### Mini Beamer Transmitter

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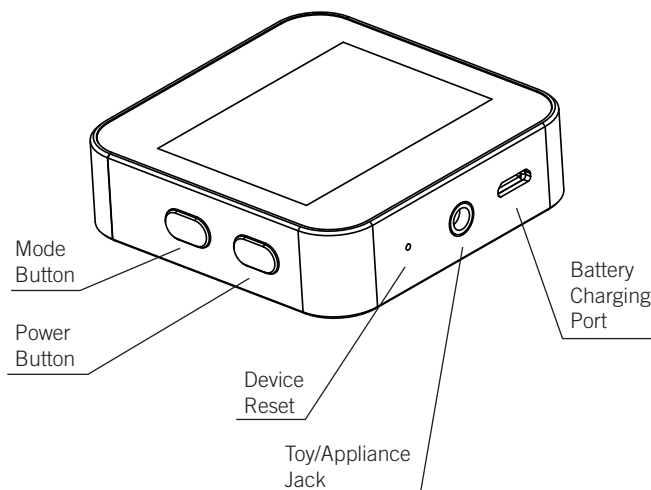
Activate power button for 5-seconds to turn wireless transmitter on or off.

Mini Beamer Transmitter must be paired to an AbleNet wireless receiver to operate. One transmitter can be paired with multiple wireless receivers to activate multiple items at the same time.

To activate, place a finger or other portion of body within 10mm of the activation surface or touch the activation surface. Indicator light will blink and an auditory beep will play upon activation. The indicator light will blink and auditory beep will play a second time upon release of the activation.

To charge, select the appropriate wall adapter for your region. Insert micro USB portion of wall charger into battery charging port on Mini Beamer Transmitter and plug wall adapter into wall outlet. While charging the indicator light will blink on and off. When fully charged the indicator light will remain lit.

### Mini Beamer Receiver

### Mini Beamer Receiver Operation

Activate power button for 5-seconds to turn wireless receiver on or off.

Mini Beamer Receiver must be paired to an AbleNet wireless transmitter and the plug-to-plug adapter must be attached to the device you are trying to operate with a switch. One receiver can be paired with up to eight wireless transmitters to allow multiple people to participate at the same time.

To enter pairing mode, hold the mode button down 5-seconds. Activate each wireless transmitter to pair it with the wireless receiver. Activate the mode button to exit pairing mode.

Select desired mode of operation by activating the mode button.

- Direct: Device is activated when wireless transmitter is activated.
- Seconds: Device is activated for 1 to 60 seconds after a wireless transmitter is activated. Activate the power button to change the seconds.
- Minutes: Device is activated for 1 to 99 minutes after a wireless transmitter is activated. Activate the power button to change the minutes.
- Latch: Device is activated and remains on when wireless transmitter is activated. Activate the wireless transmitter a second time to turn off.

To charge, select the appropriate wall adapter for your region. Insert micro USB portion of wall charger into battery charging port on Mini Beamer Receiver and plug wall adapter into wall outlet. While charging a plug symbol will flash next to the battery indicator on the wireless receiver. When fully charged the plug symbol will remain lit.

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

### Additional Information

Visit [www.ablenetinc.com](http://www.ablenetinc.com) for mounting options. Mini Beamer Transmitters and Receivers have an effective range of up to 30 feet.

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

For more information on AbleNet products and services, contact:  
AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113  
1-800-322-0956  
[www.ablenetinc.com](http://www.ablenetinc.com)

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AbleNet, Inc., 2625 Patton Road, Roseville, MN 55113