

Quickstart Guide:
iTalk2™ & iTalk2™ with Levels communicators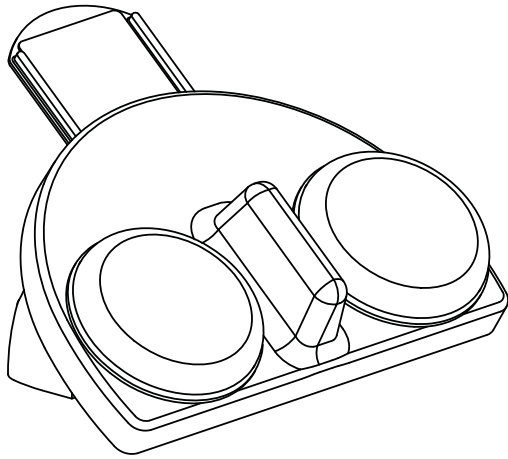


www.ablenetinc.com

1 Insert Battery and Turn On

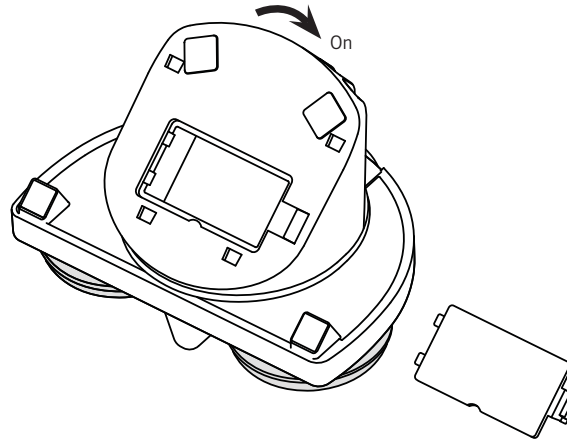Connect one 9-volt battery at the bottom of the device.
Rotate the Volume ON/OFF knob to turn the device on.


2 Enter Record Mode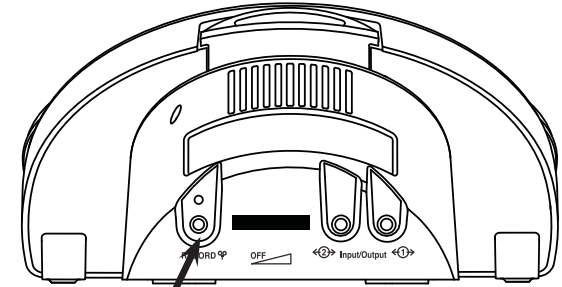

Press and Hold,
then Release

Press and Hold the RECORD button until the RECORD
Light activates, then Release.

3 Record Messages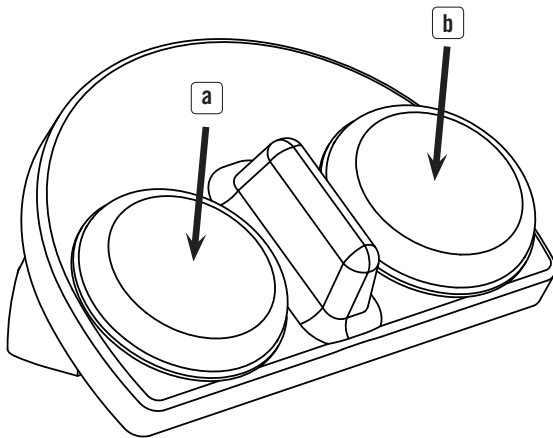


a. Press and Hold a switch top, begin speaking when
you hear a beep. Release when finished.

b. Repeat for second location.

RECORD Light will flash while recording.

4 Exit Record Mode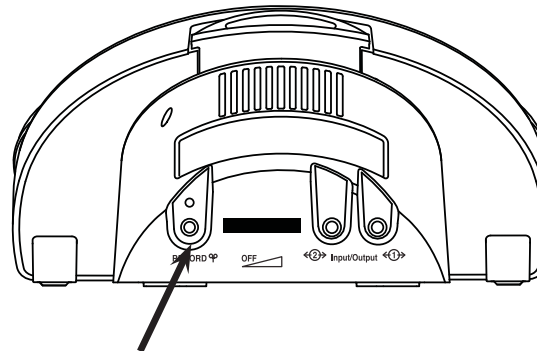


Press and Release

Press and Release the RECORD button to exit Record
Mode.


NOTE: iTalk2 has 2 minutes of total recording time and
iTalk2 with Levels has 4 minutes. The light will turn yellow
and flash when only ten seconds of time remain.

5 Play Messages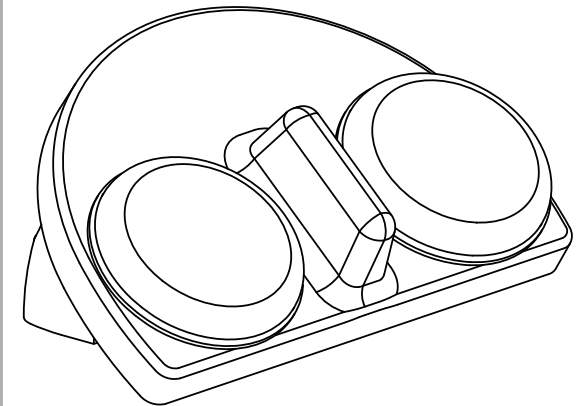

iTalk2 is now programmed and ready for use.

Advanced Features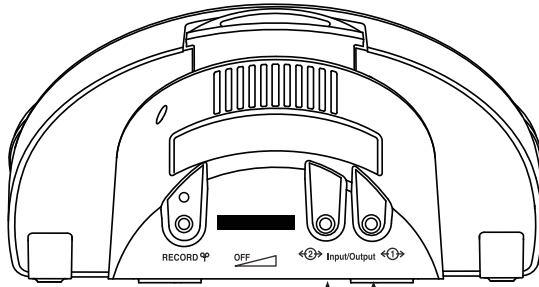

Yellow - Input/Output Jack 2

Red - Input/Output Jack 1

Controlling Toys or Appliances

The iTalk2's Input/Output jacks allow you to activate switch-adapted toys or appliances with this device.

1. Plug any switch-adapted toy or appliance into one or both of the Input/Output jacks.
2. Record a message into the corresponding location(s).
NOTE: Red switch activates jack 1, Yellow activates jack 2.
3. Press the switch top to play the message and activate the attached toy or appliance for the length of the message.

NOTE: To control an electrical appliance, a PowerLink® is required. For information about adapting toys and appliances, visit www.ablenetinc.com


Playing a message with an External Switch

iTalk2's Input/Output jacks default to the settings above. To use external switch(es) with the iTalk2:

1. Turn the device off.
2. Attach switch(es) of your choice with 1/8-inch (3.5mm) plugs into one or both of the Input/Output jacks.
3. While holding external switch(es) down, turn iTalk2 on.

The Toy/Appliance feature is disabled when using external switch(es). To revert to default Toy/Appliance function, turn the device off and back on.


Using Pictures and Symbols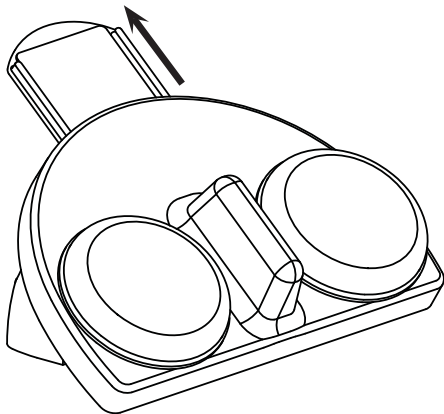

Two Snap Switch Caps have been included with the iTalk2. Use these clear plastic caps to secure pictures or symbols to the switch tops.

iTalk2 also includes a built-in symbol holder at the top of the device. Pull the clear plastic holder away from the device to store any picture or symbol up to 2 by 2-inches.

Using Levels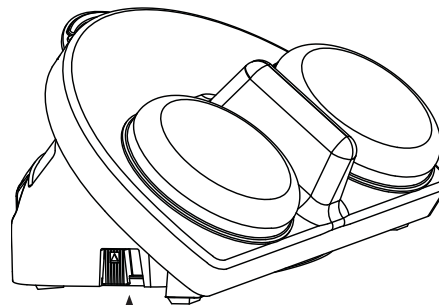


Levels Switch
(iTalk2 with Levels only)



The iTalk2 with Levels provides three recording levels for increased flexibility.

Use the toggle switch on the side of the iTalk2 to select the appropriate level. You can record separate messages for each message location on each level.

Product Map and Warranty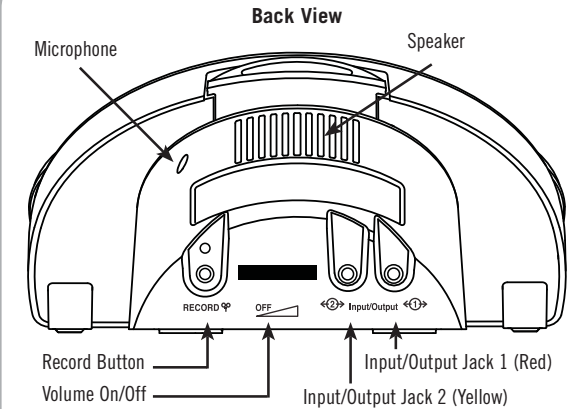

Bottom View

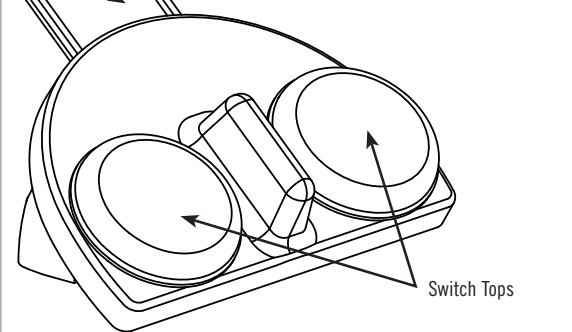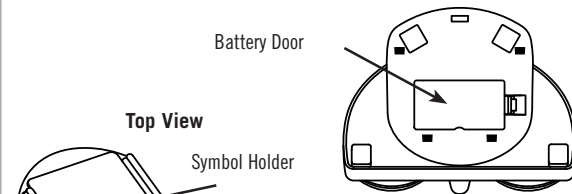Warranty/Repair/Return Information

AbleNet offers a limited one-year warranty on the iTalk2 and iTalk2 with Levels communicator products. For full warranty details, visit www.ablenetinc.com

For more information on AbleNet products and services:

AbleNet, Inc.,
Minneapolis/St. Paul, MN
800-322-0956 (US & Canada)
Internet: www.ablenetinc.com
© 2011 AbleNet, Inc., All Rights Reserved

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AbleNet, Inc., Minneapolis/St. Paul, MN
© 2011 AbleNet, Inc., Tous droits réservés