Quickstart Guide: Single Switches

AbleNet®

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AbleNet®

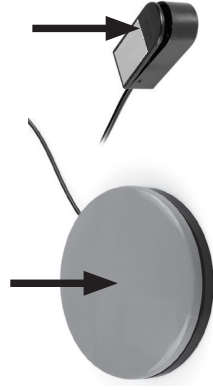
Connect Switch

Plug a single switch into any switch-adapted device with a standard, mono 3.5mm switch jack.

Note: For computer access a computer switch interface is required. (sold separately)

Press* Activation Area

Once the activation area is pressed, any attached device will activate.

*String, Grasp, and Tip switches are activated with a pull, squeeze, or tilt, respectively.

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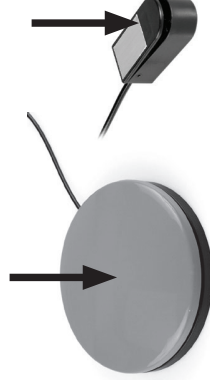
Beyond the Basics

Storage and Care
Disconnect switch before cleaning with a damp cloth or alcohol.
Do not pull or wrap the cable around the switch.

Do not immerse in water or use solvents to clean.
Do not open or attempt to take apart the switch.
Do not expose switches to extreme heat or cold.

Mounting Options
AbleNet switches can be mounted in a variety of different ways. For flat surfaces and simple mounting, try Cling™ products or Velcro®.

For complete mounting systems and connections designed just for your switch, visit our on-line store at www.ablenetinc.com. Locate your specific product's page, and click on "Related Items".

Other Features

Big Red and Jelly Bean switches:
A Snap Switch Cap (sold separately) may be used to secure a picture or symbol on the Big Red and Jelly Bean switches.

Big Red and Jelly Bean twist switches include four twist-off switch tops. Firmly, rotate switch tops counter-clockwise to remove, clockwise to attach.

Colored stickers
If your switch includes colored stickers, use them to customize its appearance.

Switch covers
If your switch includes a cloth cover, wash it by hand, using a gentle detergent when necessary.

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Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com. For more information on AbleNet products and services, contact:

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AbleNet, Inc.