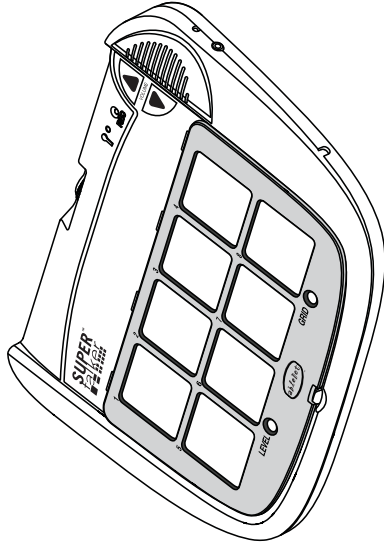
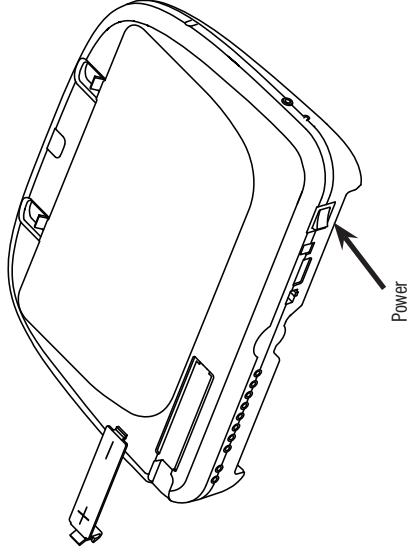
**Quickstart Guide:**  
**SuperTalker™** progressive communicator


Overlays and ideas at [www.ablenetinc.com](http://www.ablenetinc.com)

**1 Powering SuperTalker**

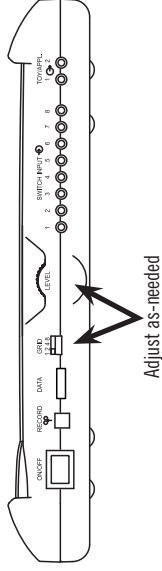Insert four AA batteries into the device as shown. Power On/Off is located on the spine of the device.


**2 Select Level and Grid**

With the desired Overlay and Keyguard in place: Select a LEVEL (1-8) and Grid (1, 2, 4, 8) using the controls on the spine of the device.


*NOTE:* Be sure to match the Grid number to the Keyguard and Overlay currently installed on the device. For information on Overlays, Keyguards and Helpful Tips, see reverse.

**3 Enter Record Mode**

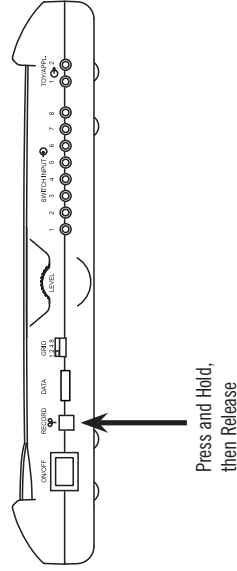Press and Hold the RECORD button until the RECORD Light activates, then Release.

**4 Record Messages**

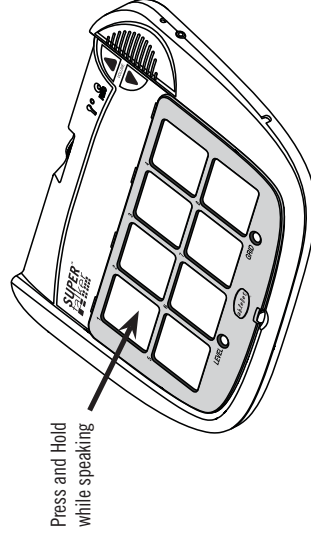
Press and Hold while speaking

Press and Hold the desired Message location, begin speaking when you hear a beep. Release when finished. Repeat for additional messages.

*NOTE:* RECORD Light will flash while recording. Speak close to the microphone for loud, clear sound.

**5 Exit Record Mode**

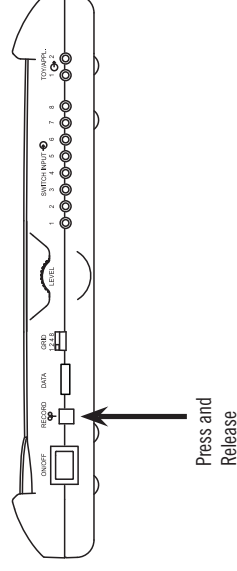
Press and Release

Press and Release the RECORD button to exit Record Mode.

*NOTE:* This device has 16 minutes of total recording time. The RECORD Light will turn yellow when only two minutes remain.

## Advanced Features
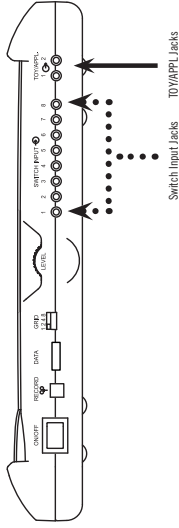
### Playing a message with an External Switch

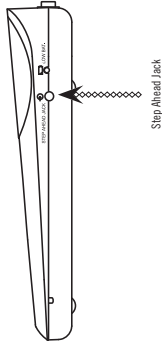
Attach the switch of your choice with a 1/8-inch (3.5mm) to the Switch Input jacks and continue to use the device as before. (Jacks indicated with ●●●●●●●●)

**NOTE:** Switch Input jacks are mapped to message locations 1 - 8 from left to right, depending on the grid used.

### Controlling Toys or Appliances

Plug any switch-adapted toy or appliance into the TOY/APPL. jack. When the SuperTalker is activated, the attached toy or appliance will activate for the length of the recorded message. (Jacks indicated with \_\_\_\_\_)

**NOTE:** TOY/APPL jacks are mapped to the first two message locations on the SuperTalker. To control an electrical appliance, a PowerLink@ 3 is required. For information about adapting toys and appliances, visit [www.ablenetinc.com](http://www.ablenetinc.com)


### Using the Step Ahead Jack

Plug the switch of your choice into the Step Ahead jack to access the messages, in-order from location 1 - 8. (Jacks indicated with ○○○○○○○○)

**NOTE:** Step Ahead Jack will move through the message locations based on the Grid selected.
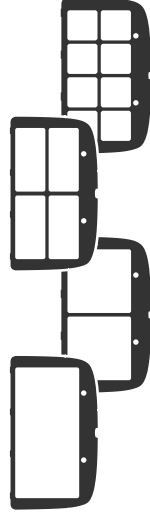
## Keyguards and Overlays

### About Keyguards

SuperTalker includes keyguards in 1, 2, 4, and 8-grid formats. Release the installed keyguard by pressing on the tab holding the bottom center of the keyguard. Small numbers at the top of each message location correspond to the External Switch jacks (see above).

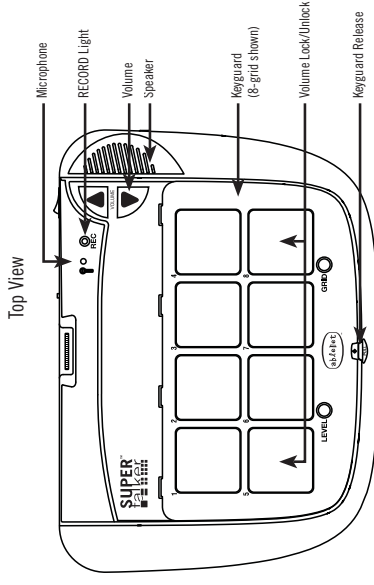
### Keyguard Storage

The keyguard storage area on the back of SuperTalker can hold up to three keyguards at once. Plain paper overlays may be stored here as well.


### Blank Overlays

Blank overlays for SuperTalker are available on-line at [www.ablenetinc.com](http://www.ablenetinc.com)


## Product Map and Warranty

Top View

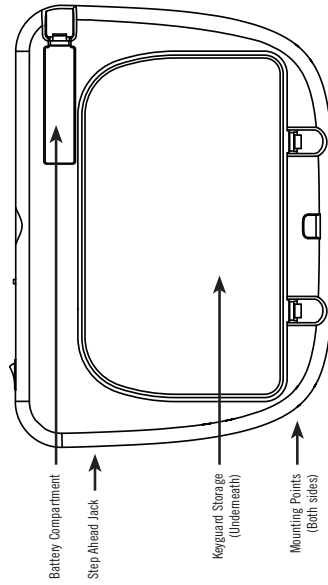Back View

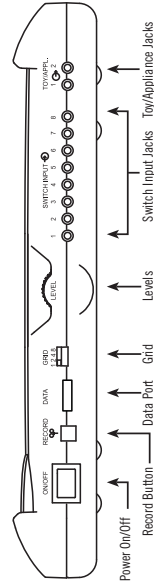Side View

### Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on the SuperTalker progressive communicator product. For full warranty details, visit [www.ablenetinc.com](http://www.ablenetinc.com)

**For more information on AbleNet products and services, contact:**

AbleNet, Inc.,  
 Minneapolis/St. Paul, MN  
 800-322-0956 (US & Canada)  
 Internet: [www.ablenetinc.com](http://www.ablenetinc.com)  
 © 2008 AbleNet, Inc., All Rights Reserved

## Helpful Hints

### Recording Tips

While in Record Mode, if the SuperTalker is not activated after 20 seconds, the device will exit Record Mode.

Master Clear - Press and hold the RECORD button for 10 seconds to delete all messages from SuperTalker.

### Playback Tips

If the Keypad in-use does not match the Grid selected, SuperTalker will not play the correct message.

If you press a message location where no message has been recorded, the RECORD Light will flash yellow.

### Volume Controls

Lock or Unlock SuperTalker's Volume controls by pressing the bottom left and right message locations simultaneously.

When adjusting volume,  
 RECORD Light Yellow – Volume unlocked  
 RECORD Light Red – Volume locked or max/min

RECORD Light Yellow – Volume unlocked

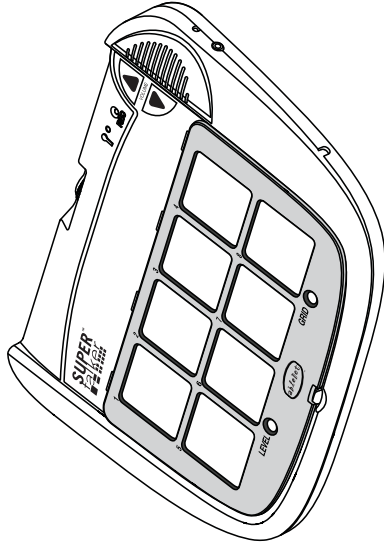
RECORD Light Red – Volume locked or max/min

### Mounting Options

A range of mounting options are available for SuperTalker at [www.ablenetinc.com](http://www.ablenetinc.com).

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© 2008 AbleNet, Inc., Todos los derechos reservados