

# Quick Start Guide

## Relax™ IR Learning Remote


www.ablenetinc.com  
1-800-322-0956

PN 98000042 (10/14 Rev. A - FN 70500204)

### Product Overview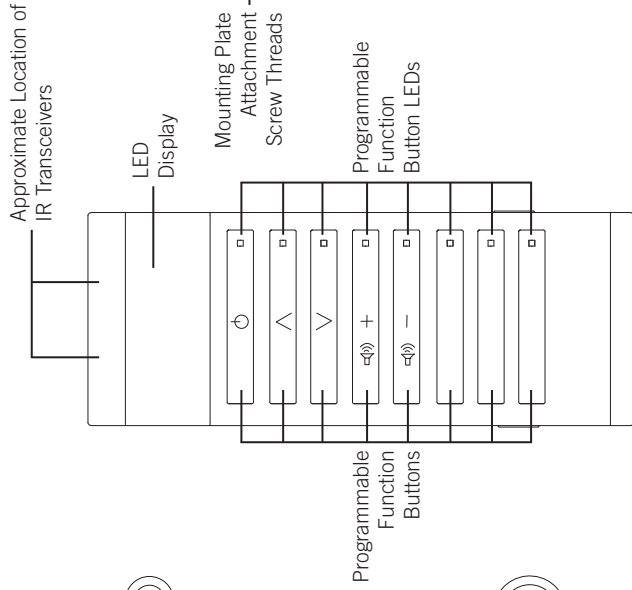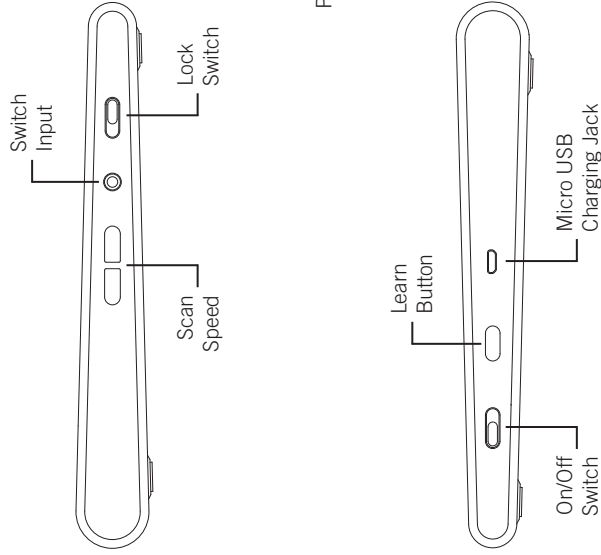

### Charging

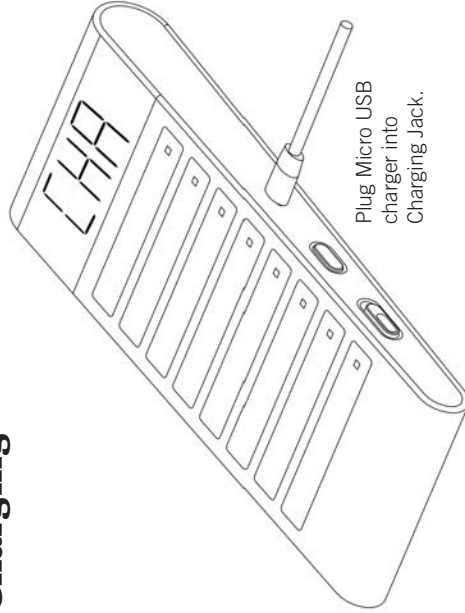
CHA = Charging  
FUL = Full Charge

### Program

Unlike some universal remotes that use a code to program the whole remote at one time, the Relax remote records the IR signal of your remote and programs one function at a time. You specify which Programmable Function Button on the Relax remote the IR signal is recorded to and can program one to eight separate functions to the Relax remote. To program your Relax remote follow these instructions:

1. Turn the Relax remote ON
2. Hold the Learn button down approximately 5 seconds until **CHA** appears on the LED Display.
3. Select a Programmable Function Button that you would like to record to. Once selected the Programmable Function Button LED will turn red.
4. Position IR remote as shown.
5. Activate desired function button on IR Remote. When the Relax remote successfully records the IR signal, the Programmable Function Button LED will turn orange.
6. To program additional Function Buttons on the Relax remote, repeat steps 3 through 5 above.
7. Once finished programming the Relax remote, press the Learn button and **CHA** will disappear from the LED Display.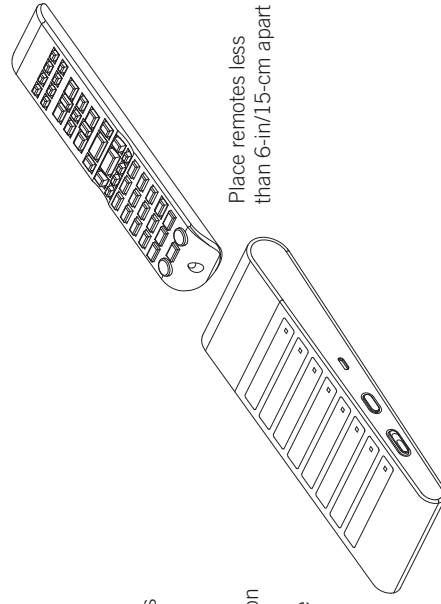

## Use

### Direct Select:

Press any of the Programmable Function buttons to activate the desired function.

**Before Switch Scanning can be used, the Relax Remote must be programmed.**

### Switch scanning set up:

1. Turn the Relax Remote on by moving the On/Off Switch from the O to the I position.
2. Using the Scan Speed + or – button, set the desired number of seconds from one to 10 for the scan rate. The scan rate is the amount of time it takes to move between Programmable Function buttons on the Relax Remote once a switch has been activated.
3. Once the scan rate has been set, plug a 3.5mm switch plug into the Switch Jack on the Relax Remote.

### Using switch scanning:

1. To start the scan, activate the switch one time. On the Relax Remote, the scan moves from top to bottom. As the scan moves through the Relax Remote, the LED on each Programmable Function button will turn on to indicate that it is the current option for the user to select.
2. To select a Programmable Function Button, the user will activate the switch one time.

It is possible for a user to activate the same Programmable Function button multiple times without having to go through the entire scan. The amount of time the scan will stay on the same Programmable Function button is determined by the scan rate that has been set. For example, if the scan rate is set to five-seconds, and the user selects a Programmable Function button for channel up, they have five-seconds to activate the switch again to select that same channel up function. After five-seconds passes, the scan will re-start at the top of the Relax Remote.

The scan on the Relax Remote will move from top to bottom for three full cycles. If no switch activation occurs after the third cycle is complete, the scanning will stop. You may then re-start the scanning by activating the switch one time.

### Single function switch set up and use:

With the Relax Remote, a user can use a switch to activate a single function on the remote.

1. Turn the Relax Remote on by moving the On/Off Switch from the O to the I position.
2. Plug a 3.5mm switch plug into the Switch Jack on the Relax Remote.
3. Activate the switch one time. You will see the scan begin at the top of the Relax Remote; the Programmable Function button LED will turn orange when the scan is on that function.
4. When the scan reaches the Programmable Function button you would like to control with the Switch, move the Lock Switch to the “locked” position.
5. Each time the switch is now activated, the single selected function will be activated and the LED Display on the Relax Remote will now show “LOC” each time the switch is activated.


**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.


## Tricks, Tips, and Warranty

### LED Display

 Digits to show scanning time. Can be from 1.0 to 10.0 seconds in 0.5 second increments.
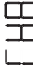
 Relax is Charging

 Function has been erased from Relax

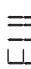 Relax is Fully Charged

 Relax battery is low

 Relax is locked into a specific location for the switch

 Relax is in Learning Mode

### Erase all programmed functions:

To erase all of the programmed functions on the Relax Remote, hold the Learn Button down for approximately 10-seconds. The LED Display will flash “dEL” off and on five times. If you press a Programmable Function Button after it has been erased, the LED will show “dEL.”

On some remotes, you may have to program a function more than once.

Size: 7.25 x 3 x 1 inch [184 x 76 x 25 mm]

Weight: 8 oz [225 grams]

Battery: Li-ion Rechargeable

Field of View: 110 degrees

Operating Temperature: 0-40°C, 32-105°F

Switch Input: 3.5mm (1/8”), mono jack

### Warranty Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

For more information on AbleNet products and services, contact:

AbleNet, Inc.  
Minneapolis/St. Paul, MN  
800-322-0956  
[www.ablenetinc.com](http://www.ablenetinc.com)

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