## Quickstart Guide: Talking Brix™ communicator

ablenet®

www.ablenetinc.com

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www.ablenetinc.com

### Programming your Talking Brix™

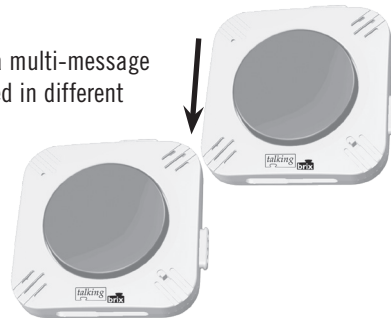
1. Turn the device ON using the power switch on back. (The device's light will flash Red 1x.)
2. Press and Hold the switch top for 5 seconds. (Begin speaking when the device's light blinks Red.) *NOTE: The device will play a portion of any recorded messages.*
3. Release switch top when finished speaking. (The device's light will turn off.)  
*NOTE: Talking Brix can store up to 10 seconds of audio.*
4. Press switch top to PLAY message. (The device's light will flash Red 1x when activated.)

To use picture symbols up to 1.25" (32mm), lift the attached Snap Switch Cap with a fingernail.

### Attaching and Mounting

Use the tabs and slots on your Talking Brix to create a multi-message communication device. Individual Brix can be arranged in different configurations to meet the specific needs of the user.

Mount any Talking Brix using the built-in magnet or place Velcro® hook and loop fastener in the recessed area on back.

### Powering the Talking Brix

When Talking Brix have a low battery, the light will blink Red 3x after message playback. Attach the Talking Brix to the included wall charger. The light will be solid RED while charging; solid Green when charged.

*NOTE: Talking Brix cannot be used while charging.*

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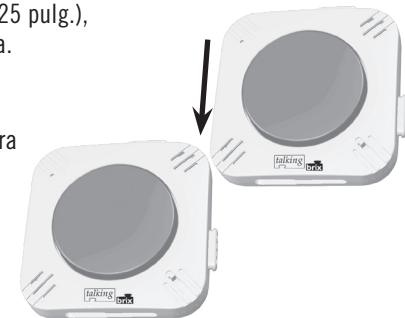
### Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on the Talking Brix communicator product. For full warranty details, visit [www.ablenetinc.com](http://www.ablenetinc.com)

### For more information on AbleNet products and services:

AbleNet, Inc., | Minneapolis/St. Paul, MN | 800-322-0956 (US & Canada) | Internet: [www.ablenetinc.com](http://www.ablenetinc.com)  
© 2009 AbleNet, Inc., All Rights Reserved

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© 2009 AbleNet, Inc., tous droits réservés
© 2009 AbleNet, Inc., Alle Rechte vorbehalten.