
## Quick Start Guide

Dual Switch Latch and Timer

www.ablenetinc.com  
1-800-322-0956

PN 9008009 (09/14 Rev. B - FN 70500212)

## Product Overview

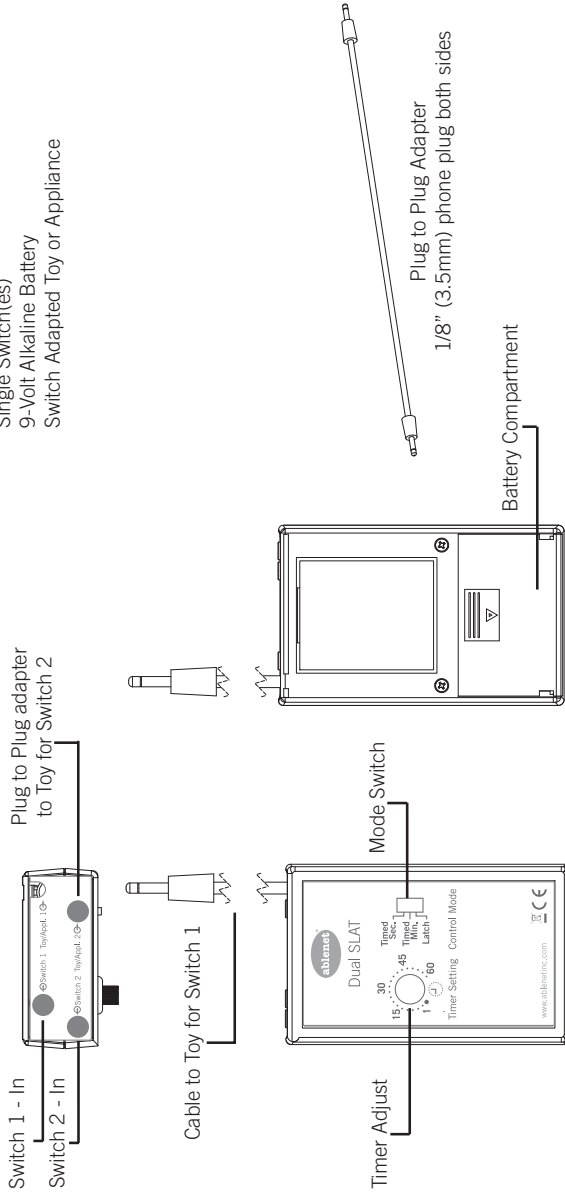**Required but not included:**  
Single Switch(es)  
9-Volt Alkaline Battery  
Switch Adapted Toy or Appliance

## Use

Most switches give switch users momentary control of their switch adapted toys or appliances. Meaning, when you press your switch, your toy or appliance will operate only for the time that you are holding down the switch. The Dual Switch Latch and Timer gives switch users more control of their devices by adding time and latch modes of control.

### Setup

1. Install 9-Volt Battery
2. Plug switch in Switch 1 - In jack or Switch 2 - In jack.
3. If your switch is plugged into Switch 1 - In jack then plug built-in cable into your switch adapted toy or appliance. If your Switch is plugged into Switch 2 - In jack then use Plug to Plug Adapter to go from the Dual Switch Latch and Timer to your switch adapted Toy or appliance.
4. Choose your mode of control using the Mode Switch.
5. Adjust the time you want your toy or appliance to operate with the Timer Adjust knob when in Timed Sec Mode and Timed Min Mode.

## Modes, Tips, and Tricks

**Timed Seconds** - The Timed Seconds mode allows for the toy or appliance to run for a preset amount of time with a single activation.

**Timed Minutes** - This mode behaves exactly like the Timed Seconds mode, but uses minutes rather than seconds.

**Latch** - One activation of a switch turns the toy or appliance on; the second activations turns it off. Use the Latch mode when you want a toy or appliance to remain on indefinitely.

You can setup two switches into the Switch Latch and timer. The switches will operate their corresponding toy or appliance independently of each other in the same mode of control.

Check battery if Switch Latch and Timer is acting erratically.

Recommend using 9-Volt alkaline batteries only.

The Timer Adjust knob shows approximately how long the toy or appliance will activate. It is not an exact indication.

## Warranty Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

For more information on AbleNet products and services, contact:

AbleNet, Inc.  
Minneapolis/St. Paul, MN  
800-322-0956  
[www.ablenetinc.com](http://www.ablenetinc.com)

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