Quick Start Guide

BIGtrack™

and

BIGtrack Switch Adapted

Drag Lock

You can easily drag an object with the BIGtrack without having to press and hold the left click button. To enable drag lock:

1. Hover the cursor over the item you would like to drag.
2. Double click the right click button to lock on to the item.
3. Move item to the desired location.
4. Release the drag lock by clicking the left click or right click button.

USB Connection

Plug the cable into your computer's USB port or a USB hub connected to your computer. You do not need to turn off your computer before connecting or disconnecting a USB device.

PS/2 Connection

1. Turn off your computer.
2. Attach the PS/2 plug adapter to the USB cable.
3. Plug the PS/2 cable into the computer's round, 6-pin mouse port.
4. Turn computer on.

Switch Use

The BIGtrack Switch Adapted has two switch ports located on the left and right side of the device. For switch use, plug a switch into each switch jack. Each switch will perform the same function as clicking the left or right button on the device.

Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

For more information on AbleNet products and services, contact:

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AbleNet, Inc.