

AbleNet®

Quick Start Guide

Chester Creek
BigBlu VisionBoard™ &
BigBlu KinderBoard™

03/19 Rev. B

About Your Keyboard (BigBlu VisionBoard Shown)

Adjustable
Feet



LED Indicator Light



Power Switch

Battery Compartment
Bluetooth Pairing Button

Step 1: Install Batteries

The battery compartment is located on the bottom of the keyboard.

1. Remove the two Phillips-head screws and set aside
2. Remove the battery compartment cover by sliding it in the direction indicated by the arrow
3. Insert two alkaline AAA batteries
4. After batteries are installed, carefully replace the battery cover and secure in place with the two screws

Step 2: Pair Bluetooth Keyboard to Device

The power switch and Bluetooth pairing button are located on the bottom of the keyboard.

1. Turn the power switch on. The LED indicator light next to the space bar will blink blue once when the keyboard is turned on.
2. Press the pairing button to enable the keyboard pairing mode. The LED indicator light will blink blue until the keyboard is paired or until 2-minutes has passed.
3. Within the Bluetooth settings on your device, select the keyboard and enter

the pairing code.

4. The LED indicator light will stop blinking after completing the pairing process.
5. Your keyboard is now ready to use.

Pair with Another Device

1. Within the Bluetooth settings on your device, select to remove or forget the keyboard.
2. Follow steps to pair Bluetooth keyboard to device.

Low Battery Alert

When the LED indicator light next to the space bar begins to flash red, it is time to replace the keyboard's batteries.

FCC

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television

Compatibility

This keyboard is designed for computers and tablets with a Bluetooth connection and Windows XP or newer. This keyboard was not tested for compatibility with macOS, iOS, Google Chrome, or Android, but may also be compatible with some devices running these operating systems via a Bluetooth connection.

reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.



Warranty/Repair/Return Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
www.ablenetinc.com
(800) 322-0956 or (651) 294-2200