AbleNet[®]


Sidekick[™] mouse adapter

www.ablenetinc.com
(800) 322-0956

Made for

- iPhone X
- iPhone 8 Plus
- iPhone 8
- iPad Pro 10.5
- iPad Pro 12.9 (2nd generation)
- iPad (6th generation)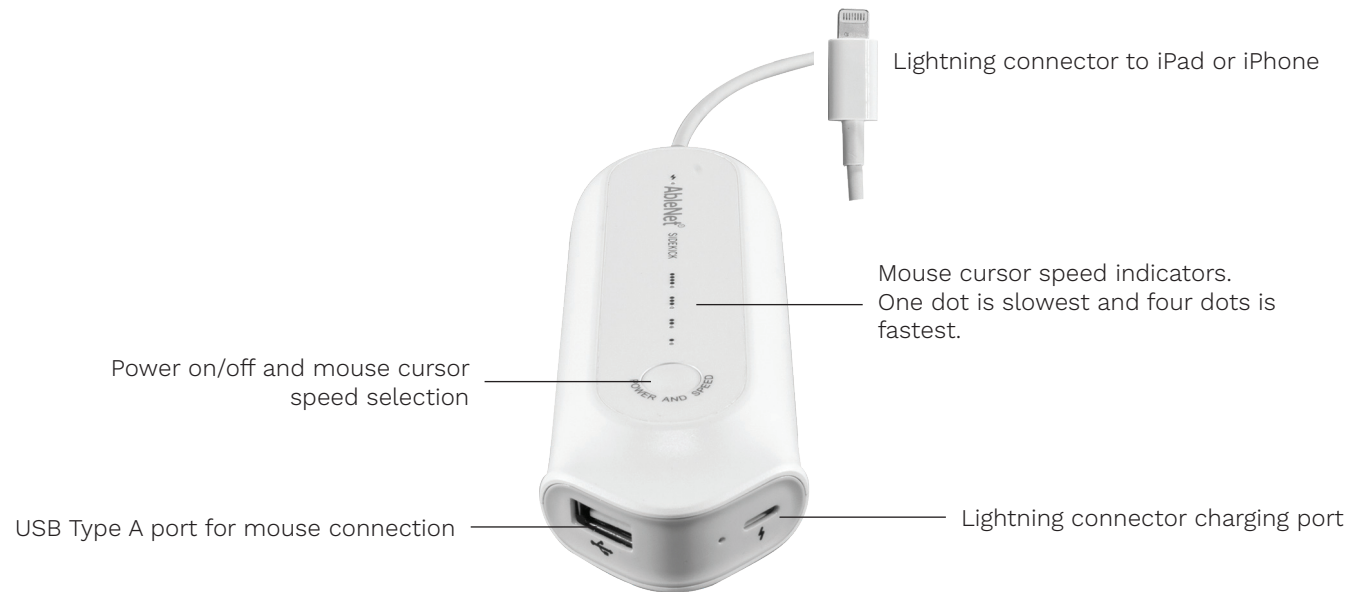

Getting Started: Two Button Mouse

1. Connect compatible USB Type A mouse to USB Type A port on Sidekick
2. Press and hold the power button on Sidekick for approximately 3-seconds. LED light will turn on.
3. Connect Lightning connector from Sidekick to iPad or iPhone.
4. Right click your mouse two times. Once the circle AssistiveTouch Pointer cursor appears on iPad or iPhone, you are ready to use your mouse.

Two Button Mouse Functions

- Left mouse click will select items.
- Right mouse click will activate the AssistiveTouch menu. The AssistiveTouch menu gives you the ability to exit an app, activate gestures, and access system commands like volume and Siri.
- Left mouse click and hold to swipe or drag items on the screen.

Getting Started: One Button Mouse

1. Connect compatible USB Type A mouse to USB Type A port on Sidekick
2. Press and hold the power button on Sidekick for approximately 3-seconds. LED light will turn on.
3. Connect Lightning connector from Sidekick to iPad or iPhone.
4. Go to Settings App > General > Accessibility > AssistiveTouch > and turn on Always Show Menu. The AssistiveTouch menu will appear on the screen (small square with dot). The AssistiveTouch menu can be moved around the screen as needed.

One Button Mouse Functions

- Left mouse click will select items.
- Left mouse click on the AssistiveTouch menu to access controls for exiting an app, activate gestures, and access system commands like volume and Siri.
- Left mouse click and hold to swipe or drag items on the screen.

Charging

- Use your existing Apple 10W or 12W wall adapter and Lightning connector cable to charge Sidekick.
- If Sidekick is connected to an iPad or iPhone while charging, both Sidekick and the iPad or iPhone will charge.

Special Use Notes

- If AssistiveTouch or Switch Control are turned on on iPad or iPhone, turn them off before connecting Sidekick.
- Turn Sidekick off before disconnecting from iPad or iPhone.
- If you have disconnected Sidekick and are unable to turn AssistiveTouch off on iPad or iPhone, turn Sidekick on and re-connect to iPad or iPhone. While Sidekick is connected, turn AssistiveTouch off.

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FCC

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

“Made for iPhone” and “Made for iPad” mean that an electronic accessory has been designed to connect specifically to iPhone or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone or iPad may affect wireless performance.

Pad, iPad Pro, iPhone, Lightning, and Multi-Touch are trademarks of Apple Inc., registered in the U.S. and other countries. iPad mini is a trademark of Apple Inc. IOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.

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