
Quickstart Guide: TrackerPro

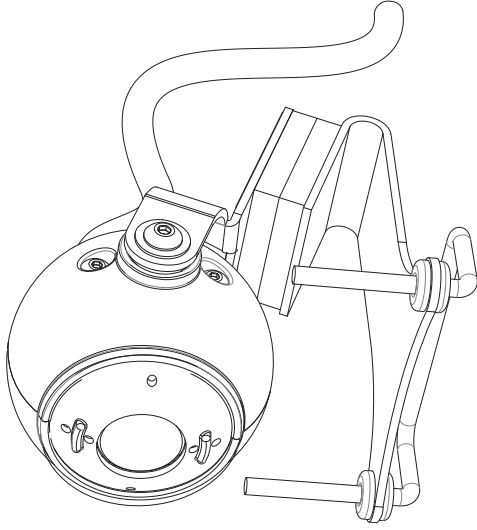
ablenet[®]

www.ablenetinc.com

What's in the Box?

1. TrackerPro
2. Mouse Click Demo Software
3. Mount/Stand
4. Dual Lock (Qty 4)
5. Switch Adapter Cable
6. USB Extension Cable
7. TrackerPro Reflective Dots - PNI2090207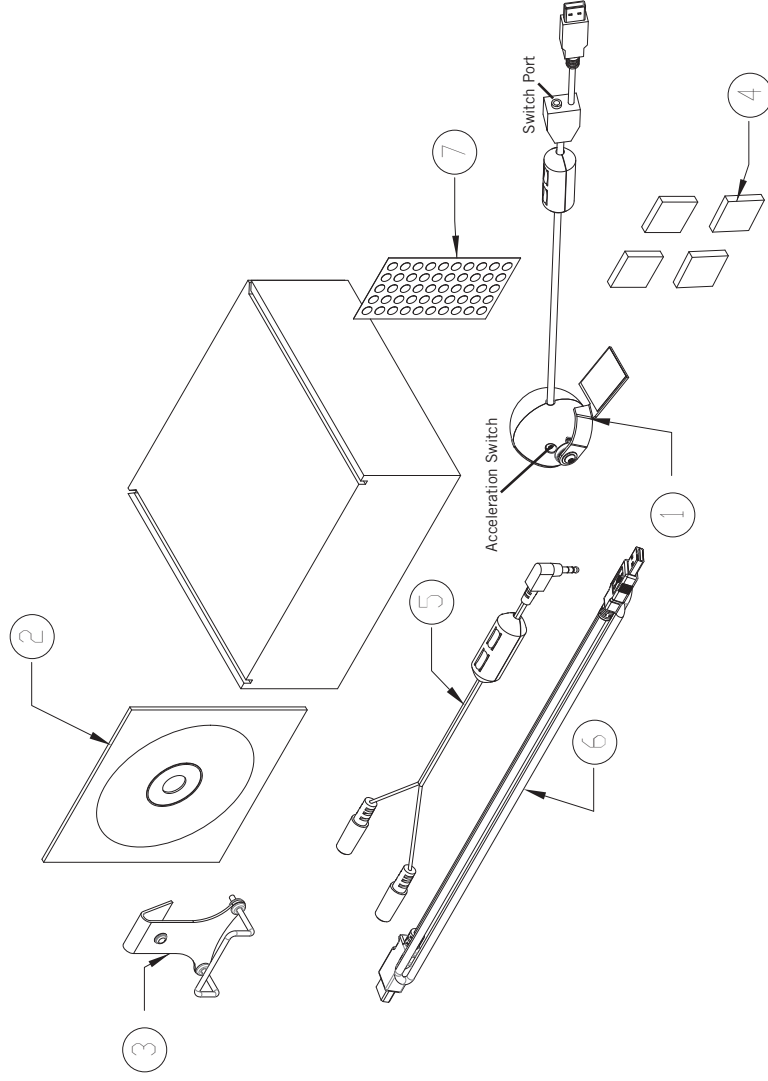

Setup

- Place Dual Lock on stand and TrackerPro Tab
- Connect TrackerPro to Stand
- Place stand on Monitor or desk/table surface.
- Plug TrackerPro into available USB port on computer
- If TrackerPro Cable does not reach – use USB extension cable (included)
- Observe White Power LED is lit (bottom LED)
- Place TrackerPro Reflective Dot on Forehead or bridge of Glasses or Baseball Cap brim
- Sit 12" – 36" from TrackerPro lens
- Observe Blue LED light (top light) when Tracker Dot is recognized by Tracker Pro

Go to www.ablenetinc.com for troubleshooting tips

Mouse Click Options

Option 1 – Software Controlled

Install included mouse click control software demo

Option 2 – Switch Use


Dual Switch - Plug Directly into Switch Port on TrackerPro

Single Switch - Plug Switch Adapter Cable into Switch Port on TrackerPro. Plug Switch(s) into Jacks on Switch adapter cable.


White Jack is Left Click

Red Jack is Right Click

Desktop Mounting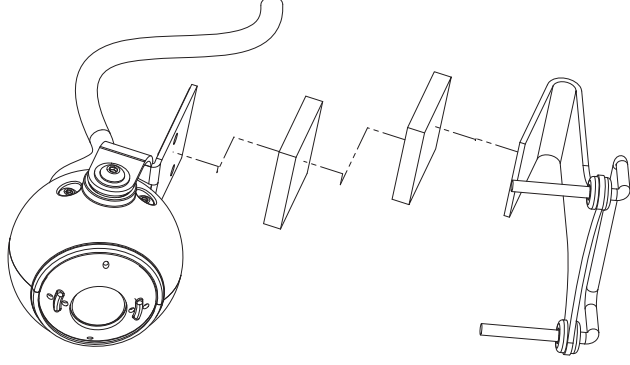

Monitor Mounting

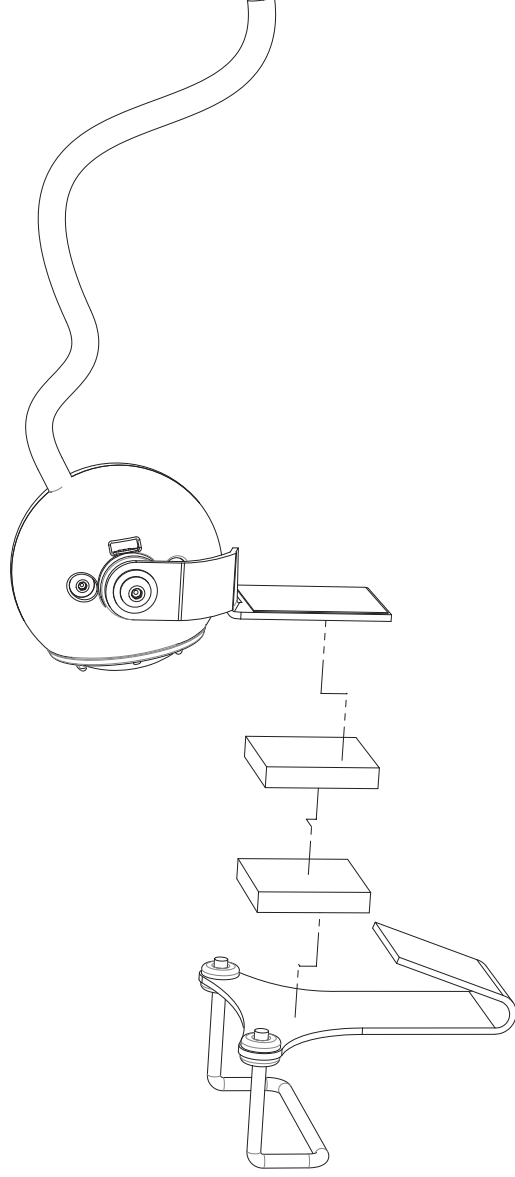Tricks, Tips, and Warranty

Size: 2" X 2" X 3.25" (5.08cm X 5.08cm X 8.25cm)

Weight: 4 ounces (0.25 lb)

Input Voltage: 5 V from USB port

Power: 1.15 watts

Field of View: 45°, symmetric

Lens: Fixed Focus

Operating Temperature: 0-40°C, 32-105°F

Infrared Wavelength Transmission: 850 nanometers

Switch Input: 3.5mm, stereo jack

Uses standard HID USB driver - Works with Windows, Mac, and Linux

Tip - If cursor is slow - move acceleration switch to 'On' position
Warranty Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com. For more information on AbleNet products and services, contact:
AbleNet, Inc.
Minneapolis/St. Paul, MN | 800-322-0956 | www.ablenetinc.com

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