
## Quickstart Guide: TrackerPro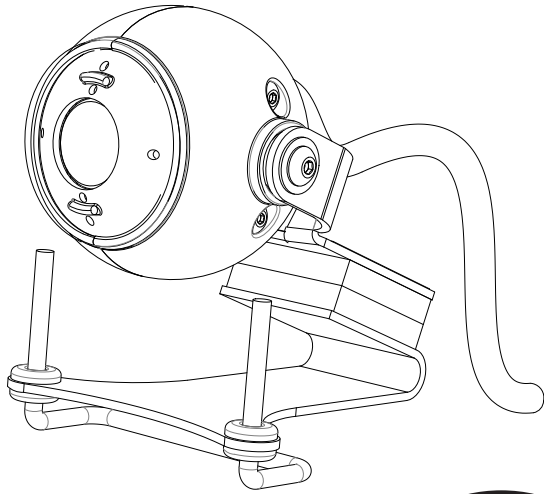

### Setup

Place Dual Lock on stand and TrackerPro Tab

Connect TrackerPro to Stand

Place stand on Monitor or desk/table surface.

Plug TrackerPro into available USB port on computer

If TrackerPro Cable does not reach – use USB extension cable (included)

Observe White Power LED is lit (bottom LED)

Place TrackerPro Reflective Dot on Forehead or bridge of Glasses or Baseball Cap brim


Sit 12" – 36" from TrackerPro lens

Observe Blue LED light (top light) when Tracker Dot is recognized by Tracker Pro

Go to [www.ablenetinc.com](http://www.ablenetinc.com) for troubleshooting tips

### What's in the Box?

1. TrackerPro
2. Mouse Click Demo Software
3. Mount/Stand
4. Dual Lock (Qty 4)
5. Switch Adapter Cable
6. USB Extension Cable
7. TrackerPro Reflective Dots - PN12090207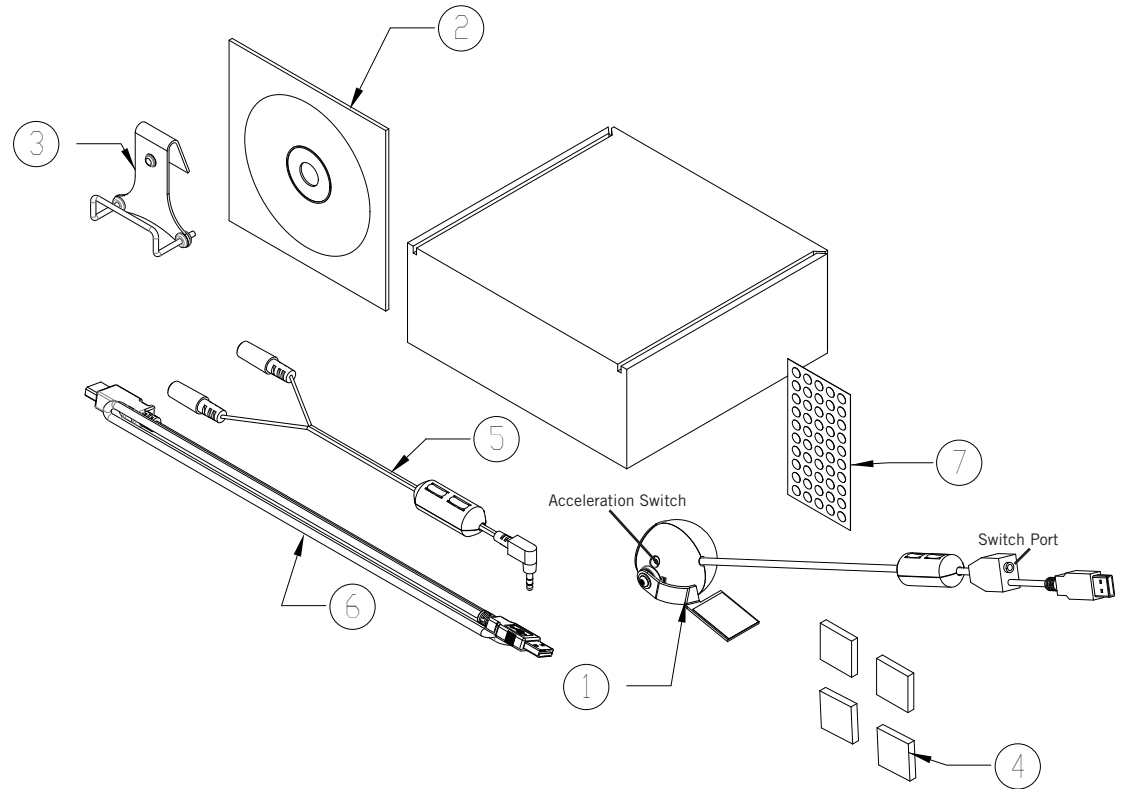

## Mouse Click Options

### Option 1 – Software Controlled

Install included mouse click control software demo

### Option 2 – Switch Use


Dual Switch - Plug Directly into Switch Port on TrackerPro

Single Switch - Plug Switch Adapter Cable into Switch Port on TrackerPro. Plug Switch(s) into Jacks on Switch adapter cable.


White Jack is Left Click

Red Jack is Right Click

## Desktop Mounting

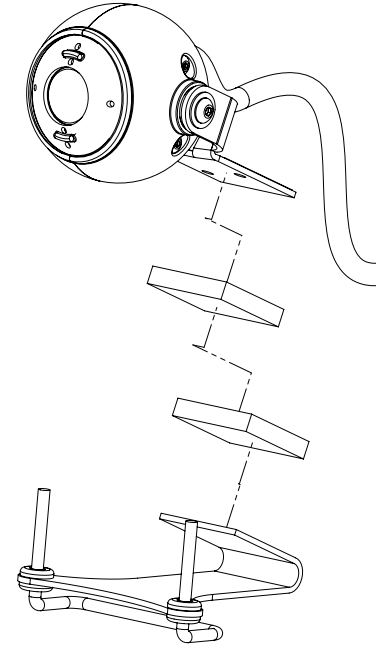## Monitor Mounting
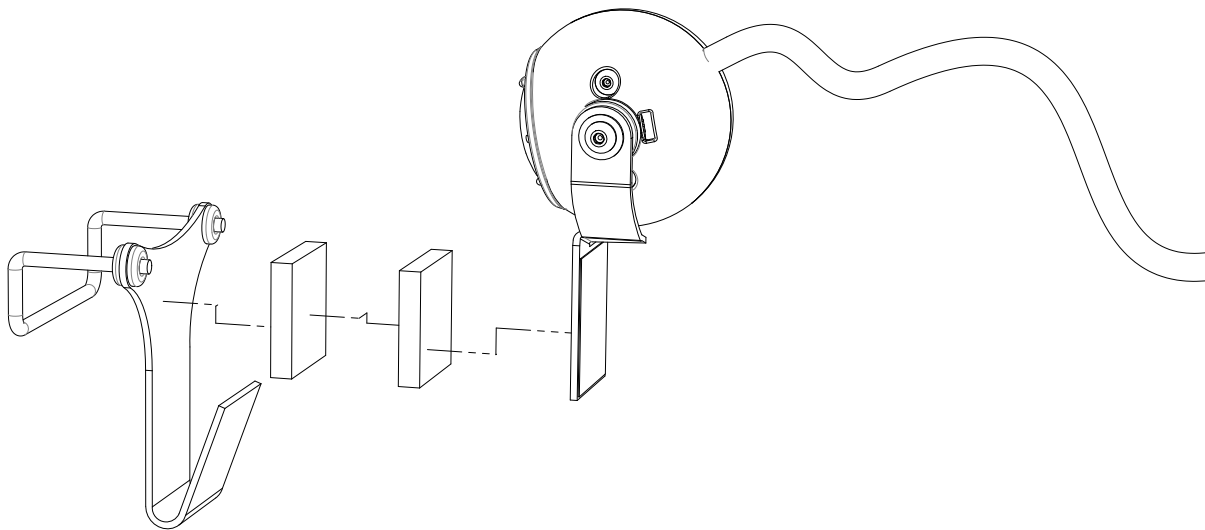
## Tricks, Tips, and Warranty

Size: 2" X 2" X 3.25" (5.08cm X 5.08cm X 8.25cm)

Weight: 4 ounces (0.25 lb)

Input Voltage: 5 V from USB port

Power: 1.15 watts

Field of View: 45°, symmetric

Lens: Fixed Focus

Operating Temperature: 0-40°C, 32-105°F

Infrared Wavelength Transmission: 850 nanometers

Switch Input: 3.5mm, stereo jack

Uses standard HID USB driver - Works with Windows; Mac; and Linux

Tip - If cursor is slow - move acceleration switch to 'On' position

### Warranty Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com). For more information on AbleNet products and services, contact:

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