
AbleNet®

Quick Start Guide

Duo™
wireless mouse

www.ablenetinc.com

Updated: July 27, 2020 7:50 AM

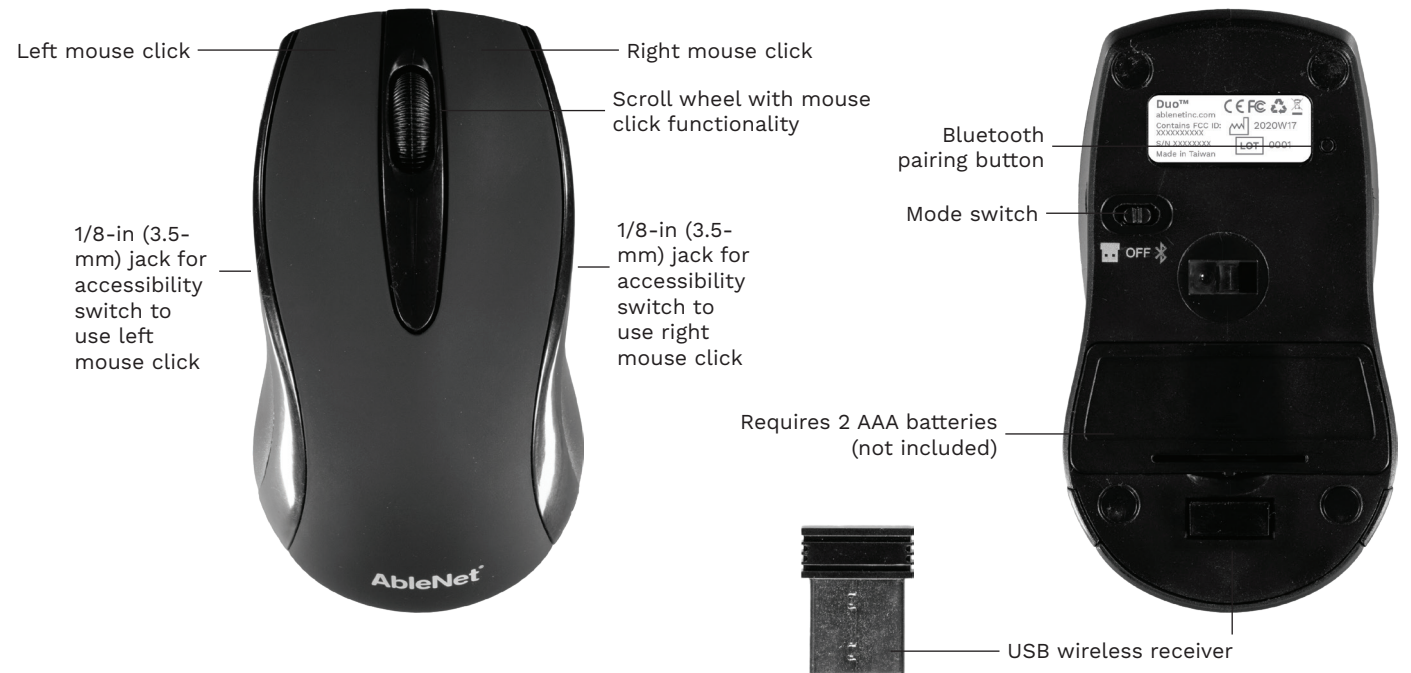Use with Bluetooth Connection

1. Remove the USB wireless receiver from the bottom of the Duo wireless mouse and store it in a safe place. The USB wireless receiver is not used with a Bluetooth connection, but would be used if a device does not have Bluetooth.
2. Open Bluetooth settings on device you are pairing mouse to.
3. Insert 2 AAA batteries into Duo wireless mouse.
4. Move Duo wireless mouse mode switch towards the Bluetooth icon to turn on Bluetooth mode.
5. Activate Bluetooth pairing button on Duo wireless mouse for 5-seconds and then release.
6. On your device, select ANDUO from the list of Bluetooth devices available.
7. Move the mouse to ensure the mouse cursor on your device moves.

Note: If storing mouse, remove batteries.

Use with USB Wireless Receiver

1. Remove the USB wireless receiver from the bottom of the Duo wireless mouse. Insert the USB wireless receiver into a USB Type A port on the device you will be using.
2. Insert 2 AAA batteries into Duo wireless mouse.
3. Move Duo wireless mouse mode switch towards the USB wireless receiver icon to turn on USB wireless mode.
4. Move the mouse to ensure the mouse cursor on your device moves.

Note: If storing mouse, remove batteries.

Use with Accessibility Switches

Connect one or two AbleNet accessibility switches with 1/8-in (3.5-mm) plugs to the Duo wireless mouse to control the left or right mouse clicks.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(800) 322-0956
(651) 294-2200
www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment (EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union (EU). If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

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FCC Title 47 Part 15b

Instructions for Use Information

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult AbleNet for help.

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