
# AbleNet®

## Quick Start Guide Dual Switch Latch and Timer

www.ablenetinc.com  
1-800-322-0956

PN 9008009 (02/20 Rev. C - FN 70500212)

### Product Overview

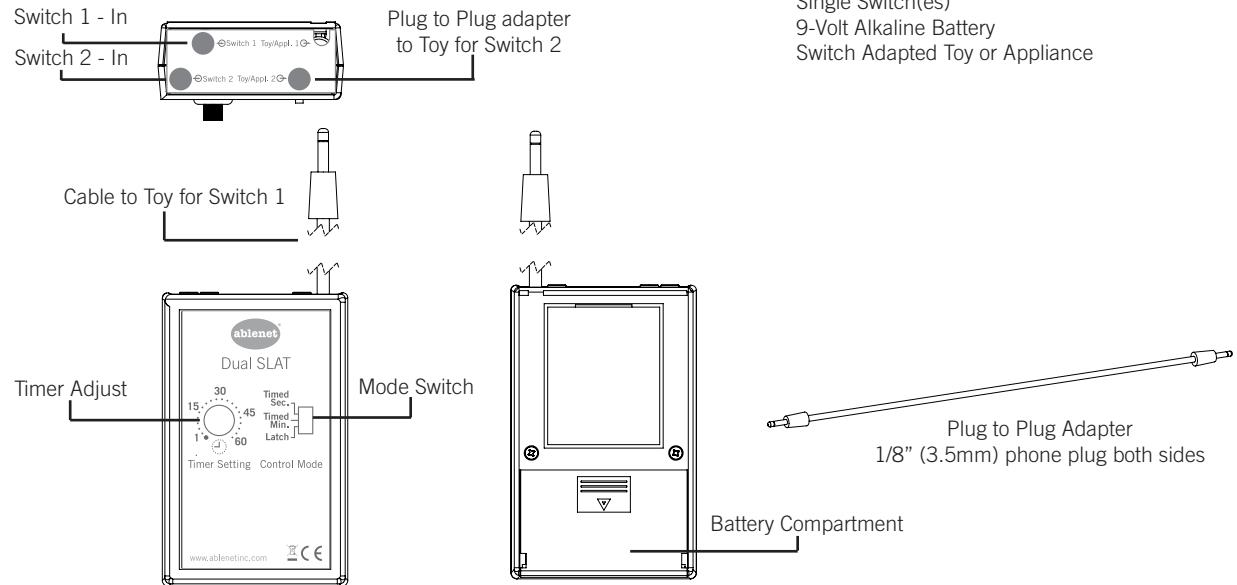### Use

Most switches give switch users momentary control of their switch adapted toys or appliances. Meaning, when you press your switch, your toy or appliance will operate only for the time that you are holding down the switch. The Dual Switch Latch and Timer gives switch users more control of their devices by adding time and latch modes of control.

#### Setup

1. Install 9-Volt Battery
2. Plug switch in Switch 1 - In jack or Switch 2 - In jack.
3. If your switch is plugged into Switch 1 - In jack then plug built-in cable into your switch adapted toy or appliance. If your Switch is plugged into Switch 2 - In jack then use Plug to Plug Adapter to go from the Dual Switch Latch and Timer to your switch adapted Toy or appliance.
4. Chose your mode of control using the Mode Switch.
5. Adjust the time you want your toy or appliance to operate with the Timer Adjust knob when in Timed Sec Mode and Timed Min Mode.

### Modes, Tips, and Tricks

**Timed Seconds** - The Timed Seconds mode allows for the toy or appliance to run for a preset amount of time with a single activation.

**Timed Minutes** - This mode behaves exactly like the Timed Seconds mode, but uses minutes rather than seconds.

**Latch** - One activation of a switch turns the toy or appliance on; the second activations turns it off. Use the Latch mode when you want a toy or appliance to remain on indefinitely.

You can setup two switches into the Switch Latch and timer. The switches will operate their corresponding toy or appliance independently of each other in the same mode of control.

Check battery if Switch Latch and Timer is acting erratically.

Recommend using 9-Volt alkaline batteries only.

The Timer Adjust knob shows approximately how long the toy or appliance will activate. It is not an exact indication.

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## Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113  
(800) 322-0956 / [www.ablenetinc.com](http://www.ablenetinc.com)

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## For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

## For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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