Quick Start Guide: BigKeys™ LX

1

For all Computers with USB:

BigKeys LX may be used on any USB 1.1, USB 2.0, or USB 3.0 port. BigKeys LX can be used in a multiple keyboard configuration with another USB or PS/2 keyboard.

- Contains all essential keys: 97 character keyset.
- Function Keys F1 – F12
- Compatible with all major software packages.
- No special software required!

Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on BigKeys communication products. For full warranty details, visit www.ablenetinc.com.

For more information on AbleNet products and services.

AbleNet, Inc.
Minneapolis/St Paul, MN
800-322-0956 (US & Canada) / www.ablenetinc.com
© 2013 AbleNet, Inc., All Rights Reserved.

2

BIGKEYS LX FEATURES

CURSOR KEYS: PgUp, End, PgDn, and Home

To use these special cursor control keys, hold down the F key and the arrow pad keys become PgUp, End, PgDn and Home.

FUNCTION KEYS: This feature lets you access the Function keys F1 - F10. Simply hold down the F key and the numeric row becomes F1 through F10.

SPECIAL CHARACTERS:

For This Character	Use These Keys	For This Character	Use These Keys
[F with ;	F11	F with /
]	F with ‘	F12	F with `
{	F with :	€	F with E
}	F with “	Prt Scrn	F with P

3

ASSIST MODE

The ASSIST MODE accommodates those who cannot press two (or more) keys simultaneously. ASSIST MODE works with and enhances the Windows “Sticky Key” Accessibility Option. When not in ASSIST MODE, the function keys are produced by pressing simultaneously the F key and a number key. (F+1 produces the F1 function, etc.)

Assist Mode enables you to press the F key and have it remain active until you press another key. In ASSIST MODE, the F key becomes a “Sticky Key”.

Examples of using ASSIST MODE:

- To obtain an F1 in ASSIST MODE, press and release the F key, then press and release the 1 key.
- To obtain an “Alt-F4” in ASSIST MODE, press and release the Alt key, then press and release the F key, then press and release the 4 key. The Windows “Sticky Keys” Accessibility Option must be turned ON for this example.

To enter ASSIST MODE press the CAPS LOCK key three times in a row. To exit ASSIST MODE and return the keyboard to normal operation, again press the CAPS LOCK three times in a row. Note that while in ASSIST MODE it is necessary to press the F key twice to send an F to the computer.


4

NO RUN-ON OPTION

BigKeys LX is designed to allow only one keystroke to be sent to the computer for each key depression. This feature prevents multiple characters that might occur inadvertently when holding down a key too long. See Set Options below to turn the No Run-On Option off.

QWERTY AND ABC LAYOUTS

BigKeys LX offers a QWERTY and ABC layout. By removing and rearranging the key caps, you can reprogram your BigKeys LX to use either layout. See Set Options below to change from QWERTY to ABC or ABC to QWERTY.

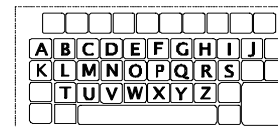ABC Layout

QWERTY Layout

SET OPTIONS

Be sure the computer is powered on and that the BigKeys is unplugged. Hold down the numeric key for the option desired. While holding down the key, plug the keyboard into a USB port on the computer. Release the key after 5 seconds, the option will now be in place.

- QWERTY Mode use numeric key 1
- ABC Mode use numeric key 2
- No-Run-On use numeric key 3
- Run-On use numeric key 4

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© 2013 AbleNet, Inc. Todos los derechos reservados.